SVENSKA

## Utprovning av mediven thrombexin 18

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## ENGLISH Measuring for mediven thrombexin 18

Correct measuring is essential for adequate thrombosis prophylaxis. **The stocking size is determined by the ankle measurement (cB).** The thigh circumference measurement (cG) confirms the selected stocking size.

Anti-embolism stockings with medically correct compression. Highly resistant to sunlight and perspiration. Skin friendly. Air permeable. Fast drying. For further information please see technical data sheet.

### Whole number measurements:

In the interests of better handling and legibility measurements for garments are given in whole centimetres. Any potential gaps that may result from this are covered by measurement and production tolerances. The internationally recognised rounding rules apply for the measurements (round down up to 4/10, round up from 5/10). This was taken into account when designing the paper measuring tapes provided by the manufacturer.

### Indications:

To promote venous return for the prevention of pre-, intra- and postoperative thromboembolism in immobilised patients confined to bed.

### Contraindications:

Anti-embolism stockings are contraindicated in: arterial circulation disorders, right-sided heart failure, massive oedema of the leg, pulmonary oedema, pre-existing gangrenous damage.

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