
Celia

COMFORT BRA

Breathable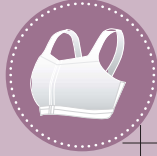

4-way stretch

COMPRESSION LEVEL 5 - LIGHT

Made in Slovakia by TYTEX AS, Huskova 21, DK-230 Hrad, www.tytext.com

www.carefix.eu

NOTE: Due to possible post-surgical swelling and edema, you may need to go up 1-2 sizes compared to your normal bra size. For more compression go down one size and for less compression go up one size. Important: Always measure yourself to find the right size!

Band Size	AA	A	B	C	M	L	XL	2XL
CM	65 cm	70 cm	75 cm	80 cm	85 cm	90 cm	95 cm	100 cm
IN	26 in	28 in	30 in	32 in	34 in	36 in	38 in	40 in
	110 cm	105 cm	100 cm	95 cm	90 cm	85 cm	80 cm	75 cm

Wash with similar colours in a washing bag. Use of chlorine-based disinfection will decrease durability and function.

Wash 100°

Colorfast 100°

Latex free


60% Cotton

25% Polyamide

15% Elastane

Art. No. 5006/5007

Available colours: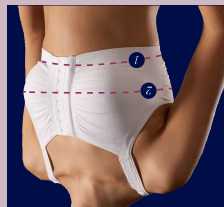

1 Measure around your body, just under your breast (Band Size)

2 Measure around your body, at the fullest part of the breast (Bust Size)

3 The difference between the Band Size and the Bust Size shows your Cup Size, example: (Bust Size - Band Size = Cup Size)

In 38 - 32 = 6 in, i.e. Cup B

10 - 12 cm / 4.0 - 4.7 in = Cup AA

12 - 14 cm / 4.7 - 5.5 in = Cup A

14 - 16 cm / 5.5 - 6.3 in = Cup B

16 - 18 cm / 6.3 - 7.1 in = Cup C

18 - 20 cm / 7.1 - 7.9 in = Cup D

20 - 22 cm / 7.9 - 8.7 in = Cup E

To find your bra size, use the Band Size and the Cup Size. If your Band Size is 80 cm/32 in and your Cup Size is B, use the Carefix size chart to find the corresponding size (size L in the chart below).

Use of chlorine-based disinfection will decrease durability and function.

NOTE: Due to possible post-surgical swelling and edema, you may need to go up 1-2 sizes compared to your normal bra size. For more compression go down one size and for less compression go up one size. Important: Always measure yourself to find the right size!

www.carefix.eu

OEKO-TEX STANDBY 100

Made in Slovakia by TYTEX AS, Huskova 21, DK-230 Hrad, www.tytext.com

TYTEX

CE

COMPRESSION LEVEL 5 - LIGHT

4-way stretch

Breathable

Celia

COMFORT BRA

carefix®

Creating functional comfort

1172535 Inserts_med IFU_113.5x470.indd 1

06/02/2018 11:51




TYTEX Customer Service:

E-mail: customer.service@tytext.com

Phone: +49 960 4700

Address: +49 960 4701

PLEASE KEEP THIS INSTRUCTION FOR FURTHER REFERENCE

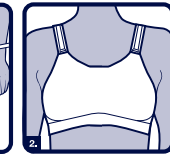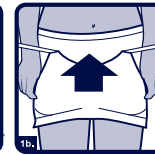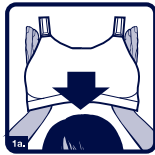EN

Indication:
Medical device for post-surgical use after breast or general surgery, when full mobility is regained. Also suitable for radiotherapy (the bra contains no metal parts).

- Instruction for use:**
1. Pull over your head (1a) or pull up like underwear (1b).
 2. Adjust the bra and ensure nothing is folded.

Precautions:
If you have any questions regarding use and correct fitting, please consult your doctor or nurse.
Make sure that bandages and/or drainage tubes underneath the bra are placed correctly to avoid pressure points and/or obstructed flow from the drainage tubes. If any unexpected discomfort or wound irritation occurs, please consult your doctor or nurse.

Care Handling:
Wash with similar colours in a washing bag. Use of chlorine based bleach or chlorine based disinfection will decrease durability and function.

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