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Date of revision of the text: 2017-11
Stand der Information: 2017-11

EN Instructions for use

Properties
Omnistrip is a sterile wound closure strip made of tan-coloured, non-woven polyamide fabric that is coated with a hypoallergenic polyacrylate adhesive.

Indications
Omnistrip wound closure strips are intended for atraumatic primary closure of minor wounds and surgical incisions (healing by first intention). When combined with sutures or staples, the strips can also be used to relieve tension at wound edges or to stabilise newly formed scars once sutures or staples have been removed.

Application
1. Peel off the backing paper in the middle of the wound closure strips.
2. Hold the edges of Omnistrip using the outer backing papers and position on the wound at right angles to the incision and press into place. It is also possible to apply the strips singly.
3. Press on the wound closure strips and remove the external protective papers one at a time.
4. Crosswise application is recommended on joints or other parts of the body where the edges of the incision are subject to stretching and tension.

Removal
When removing Omnistrip, take care to pull each end towards the centre of the wound and then lift off carefully.

Contraindications
Omnistrip is not suitable for the closure of wounds healing by secondary intention.

Special instructions
Omnistrip 25 mm x 127 mm (REF 540686) is also suitable for fixing dialysis cannulas.

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