**cs • PLEŤOVÁ VODA DANSAC**

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**en • DANSAC SKIN LOTION**

**Contents:** Skin Lotion

**Product Description:** Skin lotion for peristomal skin cleansing.

**Indications and Intended Use:** This product is an accessory to an ostomy pouching system intended to help cleanse skin around a stoma without water.

**INGREDIENTS:** Aqua, ethyl alcohol, oil of linseed, potassium hydroxide, coconut acid, perfume, propyl gallate, hydrochloric acid.

**WARNINGS**

**Precautions:** Skin around stoma must be dry before applying your ostomy product.

**☹** Single patient – multiple use. Sharing the product with multiple users creates risk of infection.

**Notes:** Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation.

In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see [www.dansac.com/authority](http://www.dansac.com/authority) or contact local distributor.

**Instructions for Use:** Clean the skin around the stoma. Dab dry before skin barrier application.

Apply skin lotion to a dry wipe and clean the skin around the stoma. Allow skin to dry before skin barrier application.

**To Dispose:** Please follow all local disposal regulations.

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