## DANSAC SKIN LOTION TISSUE

- en • Skin Lotion Tissue**
- cs • Ubrously tělové**
- da • Serviet med hudlotion**
- de • Skin-Lotion-Tuch**
- fi • Ihoemulsiopyyhe**
- fr • Lingettes imprégnées de lotion pour la peau**
- hu • Bőrápoló törülőkendő**
- it • Skin Lotion Tissue**
- ja • スキンローションティッシュ**
- lt • Skin Lotion servetėlis**
- nl • Huidreinigungsdoekje**
- no • Hudlotionsserviett**
- pl • Chusteczki nasączone płynem kosmetycznym**
- sk • Tělová voda v rúškach**
- sv • Hudlotionsservett**

www.dansac.com/contact

www.dansac.com/symbolglossary

913150

Dansac A/S  
Lille Kongevej 304  
DK-3480 Fredensborg  
+45 48 46 50 00  
www.dansac.com

dansac

dansac

**MD** Medical Device  
(as applicable).

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## en • Skin Lotion Tissue

**Contents:** Skin Lotion Tissue  
**Product Description:** Skin lotion for peristomal skin cleansing.  
**Indications and Intended Use:** This product is an accessory to an ostomy pouching system intended to help cleanse skin around a stoma without water.  
**Ingredients:** Aqua, ethyl alcohol, oil of linseed, potassium hydroxide, coconut acid, perfume, propyl gallate, hydrochloric acid.  
**WARNINGS:**

Extremely Flammable: Keep away from sources of ignition – No Smoking. Keep out of the reach of children. Use only as directed.

To help reduce the potential for infection and/or other complications, do not reuse.  
**Precautions:** For external use only. Skin around stoma must be dry before applying your ostomy product.

**Notes:** Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.dansac.com/authority or contact local distributor.

### INSTRUCTIONS FOR USE

Using the skin lotion tissue, clean the skin around the stoma. Allow skin to dry before skin barrier application.

**To dispose:** Please follow all local disposal regulations.

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