1a.

1b.

FR

eakin perform™ Sachet Gällanti

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EN

eakin perform™ solidifying agent

About this product: eakin perform™ is an antibiotic powder contained in a water-soluble sachet which turns liquid/stomach to a thick gel. eakin perform™ may help reduce leaks and noise, providing increased security and discretion.

Instructions for use: Insert one sachet of eakin perform™ into your pouch through the top opening (1a). For drainable pouches, insert sachet(s) via the bottom opening each time you empty your pouch (1b).

Warnings and cautions:

Do not eat. Keep out of reach of children. Do not expose the sachet contents to open wounds. Do not rupture the sachet. If contact with the sachet contents occurs, flush with water. Seek medical advice if skin irritation occurs.

Do not re-use. Attempted re-use may expose the user to the sachet contents with a potential for skin irritation.

Other information:

Store between 10° and 30° Celsius. Do not refrigerate. eakin™ accepts no liability for any injury or other loss that may arise if this product is used in a manner not intended by the manufacturer. Any serious incident encountered in relation to the use of this device should be reported to the manufacturer and the competent authority in the country of use. A serious incident is one involving serious deterioration in health or patient death.

EN Medical device. CZ Zdravotnické prostředky. DE Medizinprodukte. DK Medicinsk udstyr. FI Lääketeollisuuslaitteet. FR Dispositif Médical. GE სამედიცინო ხელსაწყო. IT Dispositivo medico. JP 医療機器. KR 의료기기. NL Medicijn hulpmiddel. NO Lægemiddel. PL Lek. PT Dispositivo médico. RO Dispozitiv medical. SE Medicinisk utrustning. SI Medicinska pripomočki. AU 医疗器械. CN 医疗器械. CZ Nepoužívejte zrovnom. DE Nicht wiederverwenden. DK Ved ikke genbruges. FI Älä käytä uudelleen. FR Ne pas réutiliser. GE არ უნდა გამოიყენოს. IT Non riutilizzare. JP 再利用しないでください. KR 사용하지 마세요. NL Niet hergebruiken. NO Ikke for gjentakelse. SE Återanvänd ej. SI Ni namenjane ponovni uporabi. AE لا تعيد استخدامه. Z

EN Do not re-use. CZ Nepoužívejte zrovnom. DE Nicht wiederverwenden. DK Ved ikke genbruges. FI Älä käytä uudelleen. FR Ne pas réutiliser. GE არ უნდა გამოიყენოს. IT Non riutilizzare. JP 再利用しないでください. KR 사용하지 마세요. NL Niet hergebruiken. NO Ikke for gjentakelse. SE Återanvänd ej. SI Ni namenjane ponovni uporabi. AE لا تعيد استخدامه. Z

30°C EN Temperature limit. CZ Teplotní limit. DE Lagerungstemperatur. DK Opbevaringsgrænse. FI Lämpötilan raja. FR Température maximale. GE დარღვევის ტემპის ზღვარი. IT Temperatura massima. JP 最高温度. KR 온도상한점. NL Temperatuurgrens. NO Temperaturgrense. SE Max temperatur. SI Temperatura omejevalja. AE لا تتعدى درجة الحرارة. Z

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© 2020 TG Eakin Ltd.