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# Corsinel®

Live as you like

## StomaSafe Plus Ostomy Support Belt

Double-layered with opening for stoma pouch

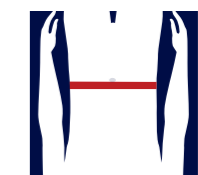
TYTEX®

www.corsinel.com

TYTEX A/S has been certified according to ISO 9001, ISO 14001 and ISO 13485. Made in Slovakia by: TYTEX A/S · Industrvej 21 · DK-7430 Ikast · www.tytex.com

SIZE	CM	IN
S/M	80 - 110	34 - 44
M/L	105 - 125	42 - 50
L/XL	120 - 140	48 - 56


Measurement at the widest abdominal circumference

Art. No. 3216  
Pc./Stk./Stück: 1  
91% Polyamide  
9% Elastane

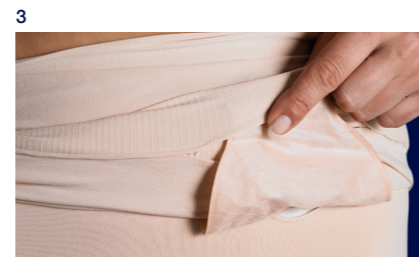Please keep this instruction for further reference

EN

Indication

Medical device for fixation and support of an ostomy pouch, used for ileostomy, colostomy or urostomy.

For comfortable wear and support. Recommended for daytime use, when more support is desired.

Instruction for Use

- Note that StomaSafe Plus is double layered. The print on the inner layer must be located on the inside at the top. Pull on StomaSafe Plus like underwear and make sure that the hole on the inner layer is positioned in front of the ostomy pouch.
- Fold up the outer layer, until the hole on the inner layer is fully visible and pull the ostomy pouch carefully through the hole.
- Fold up the ostomy pouch and place the bottom part in the small pocket at the bottom of the outer layer.
- Fold down the outer layer so that it covers the ostomy pouch.

Precaution

If too small or tight fitting, the product may affect the stoma drainage. If you have any questions regarding size, usage, indication or if discomfort or problems occur, please consult your doctor or nurse.

Wash Handling

Use of chlorine based bleach or chlorine based disinfection will decrease durability and function.

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