SI

O tem izdelku:

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EN

About this product:

eaklin® ostomy belts can be used with 1 or 2-piece ostomy systems which have bell loops.

They can be used to provide extra support around the stoma by helping to keep the baseplate in place.

Instructions for use:

After applying your eaklin® ostomy belt, the instructions attach the bell clip to one of the bell loops on the pouch or baseplate. The clip on the belt should be facing upwards to prevent rubbing against your skin. Pull the belt to ensure it is securely attached to the bell loop. Wrap the belt around the back of your waist, stretch the belt and attach the bell clip to the remaining bell loop. Pull gently to ensure it is secure. Make sure the belt is not twisted.

The belt can also be loosened after fitting. Never pull too tight so that it is uncomfortable to wear.

Other information:

eaklin® accept no liability for injury, or other loss that may arise if this product is used in a manner contrary to eaklin®'s current recommendations. Any serious incident related to the use of this device should be reported to the competent authority in the country of use. The manufacturer and one involving serious deterioration in health or patient death.

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© 2021 TG Eakin Ltd.