8 Distributör per  
B. Braun Medical NV/SA  
1831 Diegem - Belgium

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Silicone Self-Adhering Male External Catheter.

DESCRIPTION/INDICATION

The Urimed® Vision Male External Catheter is an all silicone self-adhering male external catheter designed for the management of male urinary incontinence. It is available in five sizes ranging from Small (25 mm) to Extra Large (41 mm).

CONTRAINDICATION

Do not use on irritated or compromised skin.

PRECAUTION

Do not use if allergic reaction or sensitivity occurs.

DIRECTIONS : TO APPLY

- 1) Wash penis with mild soap and warm water. Dry thoroughly.
- 2) Trim pubic hair if necessary.
- 3) Open package at perforation.
- 4) To remove plastic insert, squeeze catheter at the top of the white cone and pull to release. "Ultra" and "Short" versions only.

- 5) If uncircumcised, do NOT pull back foreskin.
- 6) Do not apply powders or creams as this may prevent a secure seal.
- 7) Unroll self-adhering catheter over penis.

- 8) Gently squeeze the catheter to properly seal adhesive to the skin.

**IMPORTANT:** Wear time may be significantly reduced if adhesive is not properly sealed to the skin.

DIRECTIONS: TO REMOVE

Gently roll catheter off the penis. Wash penis with mild soap and warm water. Dry thoroughly. Single use.

WARNING

Re-use of single-use devices creates a potential risk for the patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient.

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