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## Cleaning and sterilisation instructions for the re-use of the emteva® catheter

1. Use the catheter as instructed by your healthcare professional.
2. Once the catheter is removed rinse the catheter under the tap and place in a clean lidded container (recommend Milton solo steriliser [www.milton-tm.com](http://www.milton-tm.com))
3. Add 10mls of Milton sterilising solution (de-ionised saline/hypochlorite solution) to 150mls of water (ratio 15:1) and place in lidded container.
4. Gently swirl the container.
5. The catheter will be sterilised and ready for use in 15 minutes.
6. After 15 minutes the catheter will be sterilised, the catheter can either be removed from the fluid and stored in the original packaging, or can be left in the solution until required.
7. Before removing the catheter wash your hands, unscrew the lid and remove.
8. Rinse the catheter with tap water, the catheter is ready for re use.
9. The Milton solution in the container can be re-used for up to 24 hrs.
10. The cleaning process needs to be repeated between each catheter insertion.
11. Catheters and Milton solution should be discarded after 24 hours of use.

## A multi-use female catheter with grip lid Read the instructions carefully before using emteva® Sterile multiuse urinary catheter for intermittent self-catheterisation.

Sterilisation: Ethylene Oxide (EtO)

Manufactured on the base of corn starch 100% biodegradable, degrades to water and carbon dioxide when incinerated or composted.

Use emteva® only if prescribed by a doctor and trained nurse.

Do not use if the catheter sleeve is damaged.

Before initial use store emteva® dry and protected from sunlight.

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## A multi-use female catheter with grip & lid

Read the instructions carefully before using

Størrelse / Size CH12

 **HUNTERUROLOGY**  
caring for patients' needs

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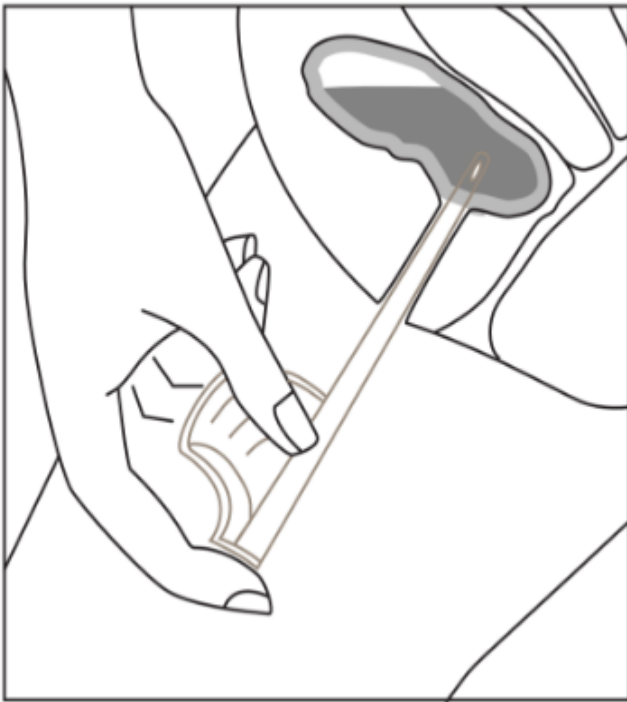
## How to use emteva®

The smooth surface of emteva® allows use without water or lubricant. Follow the steps below.

1. First wash your hands thoroughly.
2. Then wash around the urethral opening if it is the first emptying of the day.
3. Open the emteva® peel pouch at the grip end of the catheter by pulling the pouch sides apart. Take emteva® out but avoid touching the part of the catheter which is to be inserted into the urethra.
4. Press the lid down to close it tight.
5. Hold the emteva® grip between the thumb and third finger.
6. Spread the vaginal labia and gently insert emteva® into the urethra.
7. Press the lid release towards the grip with your index finger when you are ready to empty your bladder
8. Move the catheter a little to the sides and pull it out slowly to ensure complete emptying of the bladder.
9. Emteva® should be cleaned and sterilised after use (see attached cleaning instructions. Emteva® should be discarded with normal household waste after 24 hours use.

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Blod i urinen

## How often should I use emteva®?

This depends on how much fluid you drink but most people need to empty their bladder every 4-6 hours when they are awake.

## Potential side effects

A small amount of blood in the urine may occur, especially in the beginning due to irritation of the mucus membrane of the urethra.

## Contact your doctor

Contact your doctor if you experience symptoms of a urinary tract infection:

Fever

Discomfort or pain when urinating

Frequent need to urinate

Blood in the urine