UK/AU

Single Use Hydrophilic Dilation Catheter

1. Wash your hands thoroughly.
2. Peel the packet open at the luminal end and fill with water (at home from the cold tap, in hospital sterile water or saline).
3. Remove the sticker at the top of the pack to reveal the self-adhesive patch and attach the pack to a dry surface.
4. Soak the catheter for at least 30 seconds whilst preparing yourself for catheterisation.
5. Remove the catheter from the packet.
6. Carry out the catheterisation procedure as instructed. Dispose of the catheter sensibly after use.

NB: LoFric® Dila-Cath™ does not have drainage eyes and, therefore, is not suitable for bladder emptying. LoFric® Dila-Cath™ is for single use only. www.wellspect.co.uk www.wellspect.com.au

US

Instructions

Single Use Hydrophilic Dilation Catheter

1. Start by washing your hands.
2. Open the pack carefully. Pull the tabs open about 2 inches. Pour water directly into the pack. Use sterile water or as directed by your healthcare provider.
3. The pack has a self-adhesive patch. Remove the blue tab and attach the pack to a sink or other convenient, dry surface.
4. Let the catheter soak for at least 30 seconds while you are getting ready to catheterize.
5. The catheter is then ready to be inserted.

6. Carry out the catheterisation procedure as instructed. Dispose of the catheter sensibly after use.

NB: LoFric® Dila-Cath™ does not have drainage eyes and, therefore, is not suitable for bladder emptying. LoFric® Dila-Cath™ is for single use only. www.wellspect.us

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