Portex® Single Use Tracheostomy Tube Holder INSTRUCTIONS FOR USE Ref: 100/503/200

With the tracheostomy tube connector pointing towards you:

- 1 Take the holder and ensure that the 'Portex®' logo is facing away from you.
- 2 Open out one of the narrow Velcro straps (A), and thread it all the way through the back of the tracheostomy flange (B) through the left-hand hole.
- 3 Fold the narrow strap (A) back and secure to the wide band material.
- 4 Pass the holder behind the patient's neck and repeat 2 and 3 for the right-hand side.
- 5 Adjust the holder using the wide Velcro strap (D) to fit the patient, ensuring the holder is securely joined.

One size fits all adults.

Do Not Reuse: Medical devices require specific material characteristics to perform as intended. These characteristics have been verified for single use only. Any attempt to re-process the device for subsequent re-use may adversely affect the integrity of the device or lead to deterioration in performance.

Portex design mark and Smiths Medical design mark are trademarks of the Smiths Medical family of companies. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. All other names and marks mentioned are the trade names, trademarks or service marks of their respective owners.

© 2019 Smiths Medical family of companies. All rights reserved.

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Portex designmerke og Smiths Medical designmerke er varemærker som tilhører Smiths Medical-gruppen av selskaper. Symbolet ® angir at varemærket er registrert ved U.S. Patent and Trademark Office og i visse andre land. Alle andre navn og merker som er nevnt, er varemærker, handelsnavn og servicemærker som tilhører de respektive eiere. © 2019 Smiths Medical-selskaper. Med enerett.
Smiths Medical ASD, Inc.