**ARGON MEDICAL DEVICES, INC.**  
1445 Flat Creek Road Athens, Texas 75751 USA  
Tel: +1 (903) 675-9321  
Tel: +1 (800) 927-4669  
www.argonmedical.com


**EMERGO EUROPE**  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands  
+31 70 345 8570

**Argon Medical Devices UK Ltd**  
Eastgate Business Centre  
Eastern Avenue  
Burton-on-Trent  
Staffordshire  
DE13 0AT

IFU9015/0321A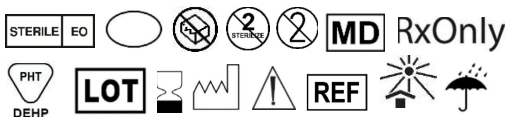

<https://www.argonmedical.com/resources/product-information>

<b>English</b>	The symbols glossary is located electronically at <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Bulgarian</b>	Речникът на символите е наличен в електронен вид на адрес <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Czech</b>	Slovníček symbolů je elektronicky umístěn na <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Danish</b>	Symbolforklaringen findes elektronisk på <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>German</b>	Ein elektronisches Symbolglossar finden Sie unter <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Greek</b>	Το γλωσσάρι των συμβόλων παρέχεται ηλεκτρονικά στη διεύθυνση <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Spanish</b>	El glosario de símbolos puede consultarse electrónicamente en <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Estonian</b>	Sümbolite seletuste kogu on elektrooniliselt saadaval aadressil <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Finnish</b>	Elektroninen symbolisanasto on osoitteessa <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>French</b>	Le glossaire des symboles est disponible au format électronique à l'adresse <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Croatian</b>	Pojmovnik simbola nalazi se u elektroničkom obliku na <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Hungarian</b>	A szimbólumok összefoglalása elektronikusan itt található: <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Italian</b>	Il glossario dei simboli è disponibile in formato elettronico all'indirizzo <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Lithuanian</b>	Simbolių žodyno elektroninė forma pateikta <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Latvian</b>	Simbolu glosārijs ir atrodams elektroniskā veidā vietnē <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Dutch</b>	Uitleg over de symbolen is elektronisch beschikbaar op <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Norwegian</b>	Symbolforklaringen befinner seg elektronisk på <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Polish</b>	Glosariusz symboli w postaci elektronicznej dostępny jest pod adresem <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Portuguese</b>	O glossário de símbolos está localizado eletronicamente no endereço <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Romanian</b>	Glosarul simbolurilor este stocat electronic la <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Slovak</b>	Glosár symbolov je v elektronickej podobe k dispozícii na stránkach <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Slovenian</b>	Slovar simbolov se nahaja v elektronski obliki na <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Swedish</b>	Symbolforklaringen hittas elektroniskt på <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Turkish</b>	Semboller sözlüğü elektronik ortamda <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a> adresinde yer almaktadır

## EN - ENGLISH

### Connectors/Stopcock

**Intended Use/Purpose:** The product are intended for used in conjunction with drainage bags and channeling catheter for fluids.

**Devices Description:** This product are stand-alone components built to user specifications. The device offers wide range of connecting tube lengths, various types of adapters and stopcocks for all types of drainage bags.

**Indication for Use:** Connectors are used for channeling blood, urine and other body fluid between catheters and all types of drainage bags.

#### Warnings

- Contents are supplied sterile and are intended for single use only. Do not re-sterilize, reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury.
- Inspect the package integrity before use.
- Do not use if package is open or damaged and if the expiry date has been exceeded.
- Do not continue to use if any of the component are damaged during the procedure.

#### Precautions

- This device is intended for use by trained, qualified licensed physician.

#### Direction for Use:

- Identify the male and/or female ends of connectors.
- Insert male and female luer locking ends together.
- Twist to confirm secure connection.

#### Possible Complications:

There is no known complication related to this device.

**Storage:** Store at controlled room temperature.

**Disposal:** After use, this product may be a potential biohazard. Dispose in accordance with applicable laws and regulations.

**NOTE:** In the event a serious incident related to this device occurs, the event should be reported to Argon Medical at [quality.regulatory@argonmedical.com](mailto:quality.regulatory@argonmedical.com) as well as to the competent health authority where the user/patient resides.

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