ENGLISH

 Please read the instructions for use thoroughly before using the product

Sterile CerviLube™

Sterile Lubricating Jelly

Carbomer, Paraben & CHG free

Validated by Hologic for use with the ThinPrep® Pap test when used as instructed

(see www.hologic.com/thinprep for more details)

For Hospital and Professional Use

Use: Intended for medical purposes to lubricate body orifices and to facilitate entry of diagnostic or therapeutic devices.

Directions:

- For packet: Tear packet. Squeeze to obtain desired amount. Apply as needed.
- For Tube: Remove cap. Pierce the seal with the spike on the cap. Squeeze the tube to obtain the desired amount. Apply as needed.

Warnings: Keep out of eyes and ears. If irritation or discomfort occurs, discontinue use and consult a physician. Keep out of reach of children.

Contains: Water (aqua), glycerin, propylene glycol, hydroxyethylcellulose, sodium benzoate, polysorbate 20, disodium EDTA, lactic acid, sodium hydroxide

Tamper Evident:

- **For packet: Do not use if the packet is torn or cut.**
- **For Tube: Do not use if the seal is broken or if the tube is damaged.**

Other Information: Store at room temperature. Avoid excessive heat and humidity.

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