





KIBODAN

4 in 1 Pill Crusher

FR Coupe comprimés 4 en 1
ES Cortador de píldoras 4 en 1
NL 4 in 1 Pillensnijder
DK 4 i 1 Pilleknuser
DE 4 in 1 Pillenzerkleinerer
PT Cortador de comprimidos 4 em 1
IS 4 í 1 Töfludeilara



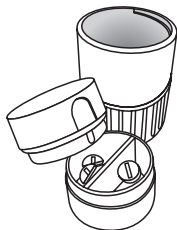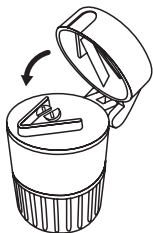

GB: Open the lid and put the pill into the correct position, then close the lid and press down hard

NL: Open het deksel en leg de pil op zijn plaats, sluit het deksel en druk er hard op

FR: Ouvrez le couvercle et mettez le comprimé dans la bonne position, puis refermez le couvercle et appuyez fort dessus.

DE: Öffnen Sie den Deckel, platzieren Sie die Pille in der richtigen Position und schließen Sie den Deckel mit Druck

PT: Abra a tampa e coloque o comprimido na posição correcta; em seguida, feche a tampa e pressione com força

ES: Abra la tapa y ponga la píldora en la posición correcta. A continuación, cierre la tapa y apriete con fuerza hacia abajo

IS: Opnaðu lokið og settu töfluna í rétta stöðu, lokaðu þá lokinu og ýttu fast niður

DK: Åben låget og placér pillen som vist. Luk låget og pres hårdt til

GB: Put the pills and split pills into the compartment

NL: Leg de hele en gehalveerde pillen in de vakjes

FR: Mettez les comprimés entiers et coupés dans les compartiments correspondant

DE: Geben Sie die Pillen und die zerkleinerten Pillen in die Fächer

PT: Coloque os comprimidos inteiros e partidos no compartimento

ES: Coloque las píldoras enteras y partidas en el compartimiento

IS: Settu töflurnar og þeim sem er búið að skera inn í hólfíð


DK: Læg de hele piller samt de knuste i hver deres rum

Keep away from children

Keep fingers away from blade



Kibodan A/S
Kronalvej 7
DK-2610 Roedovre
Lot No. 106258

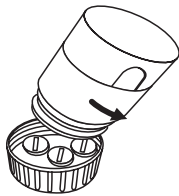

GB: Unscrew the bottom of the unit and put pills into the compartment. Screw back on completely.

NL: Draai de bodem van het apparaatje los en leg de pillen in het vakje. Draai de bodem weer helemaal vast

FR: Dévissez le fond de l'appareil et mettez les comprimés dans le compartiment. Revissez à fond

DE: Lösen Sie den Boden der Einheit und geben Sie die Pillen in das Fach. Schrauben Sie vollständig wieder an

PT: Desaperte a parte inferior da unidade e coloque comprimidos no compartimento. Volte a apertar totalmente

ES: Desenrosque la base del dispositivo y ponga las píldoras en el compartimiento.

Vuelva a enroscar la base completamente.

IS: Skrúfaðu botninn af hlutum og settu töflurnar í hólfíð. Skrúfaðu botninn aftur á

DK: Skru bunden af og placér pillerne i skruelåget. Skru låget tilbage og pillerne knuses

