IFU, Front: 280 mm x 215 mm

KIDM

KD-PENOFINE S Safety pen needles

List of compatible pens

Owen Mumford	AutoPen®	✓
	AutoPen® 2	✓
DONGBAO	YpsoPen®	✓
	NovoPen® 3	✓
Novo Nordisk	NovoPen® 4	✓
	NovoPen® Junior	✓
	NovoMix® 30 FlexPen®	✓
Ypsomed	YpsoPen®	✓
BD Medical	Puregon Pen®	✓
Eli Lilly	Luxura HumaPen®	✓
	HumaPen®	✓
	HumaPen® Ergo	✓
	Saline KwikPen™	✓
	Luxura HD HumaPen®	✓
Sanofi Aventis	HumaPen® Ergo II	✓
	OptiPen® Pro 1	✓
	TactiPen®	✓
Lantus	Sanofi Aventis Lantus	✓
	Lantus® SoloStar®	✓
Gan & Lee	GanLee Pen®	✓

EN Do not use if package is damaged

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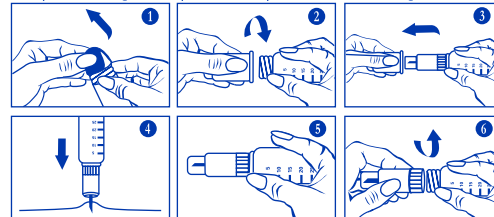
GB Safety Pen Needle

KD-PENOFINE S sterile safety pen needles for single use are intended to be connected to compatible Insulin pens for insulin administration within the frame of a diabetes mellitus treatment regimen. The safety mechanism of KD-PENOFINE S covers the needle after use. In the activated position, the safety shield guards against accidental needle-stick injuries during handling of the used product and its disposal.

Instruction for use:

1. Peel of the sealing strip (see Fig.1);
2. Attach the product to a compatible Insulin Pen by screwing it clockwise onto the pen (see Fig.2);
3. Remove the outer sheath (see Fig. 3), visually inspect whether the needle point is intact. Do not use the device if the needle tip is bent or deformed;
4. Perform a priming test prior to use to remove any air and to ensure correct dose administration. Always keep backup pen needles as a replacement. Make sure that the injection dosage is set correctly. Use the injection technique recommended by your doctor for administration. Hold the Insulin pen in a 90° angle to the skin of the insertion site and insert the needle vertically. Once the pen needle is fully inserted, press the injection button all the way down, being careful not to change the angle of the insulin pen. Keep pressing down the injection button and make sure the needle remains inserted for at least 6 seconds before removing the needle. (see Fig.4);
5. After removal, the safety protective mechanism will activate automatically to shield the needle point (see Fig.5);
6. Rotate the pen needle anticlockwise to detach it from the Insulin pen (See Fig.6). Dispose of the pen needle after single use.

Warnings, precautions:

- Read instructions before use. KD-PENOFINE S should be used according to these instruction for use and in accordance with the instructions for use of the used Insulin pen.
- Consult your doctor regarding the most suitable needle size and injection technique to be used.
- People with reduced mental /physical capacities should be assisted while using the device.
- Check the compatibility between the pen needle and the Insulin pen before use. A list of compatible Insulin pens is provided in the table.
- KD-PENOFINE S is sterile and non-pyrogenic within shelf life if the individual package has not been opened or damaged.
- Visually inspect and carefully check the device and packaging before use. Improper transport and handling may cause structural and/or functional damage to device or packaging.
- Do not use if the product or individual package is impaired, damaged, opened or contaminated.
- Do not use after the expiry date.
- Use the product immediately after opening the individual package.
- Sterilized using ethylene oxide.
- KD-PENOFINE S must not be re-used.
- Re-use of single-use devices creates a potential risk for the user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient.
- Never store the Insulin pen with a mounted pen needle as the mounted needle provides an open channel to the Insulin medication inside the cartridge – the Insulin medication may be altered (e.g. crystallization) or contaminated, air may enter the cartridge and the dosage might become inaccurate.
- Dispose of after single use according to local regulations.
- Store in a cool and dry place.
- Do not expose to heat or direct sunlight.

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