
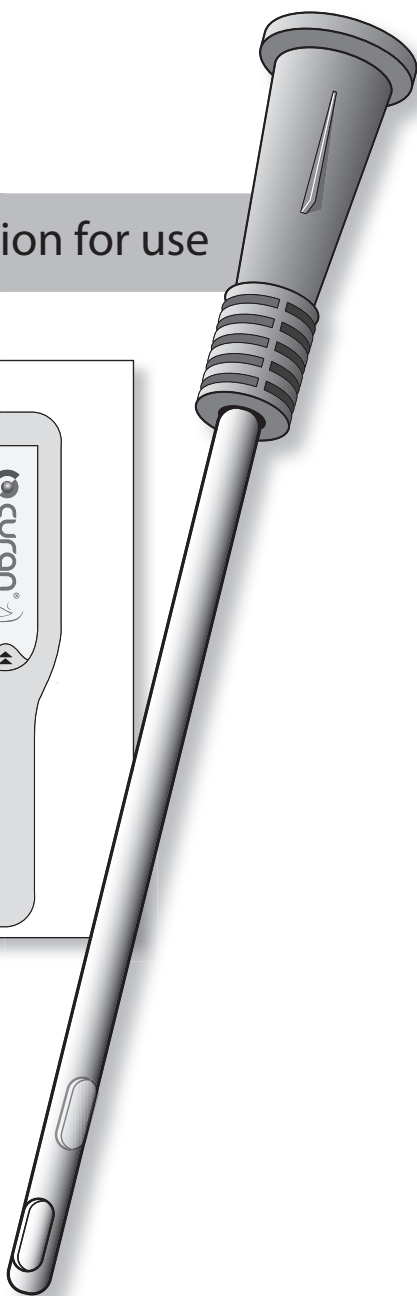
Curan Advantage[®]

Female coated catheter
for single use intermittent catheterisation.

Instruction for use

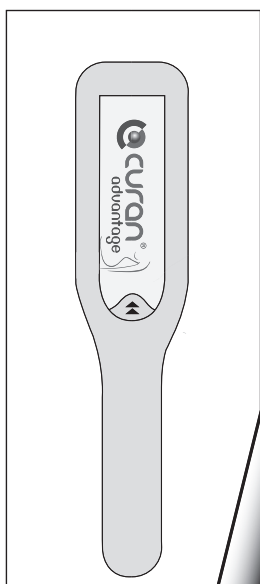STERILE R

FOR SINGLE USE

1639

PVC-FREE

Intended use

1. Single use hydrophilic coated catheter for intermittent self catheterisation for emptying of the bladder.
2. Device is ready to use.
3. For use by an individual or in a healthcare environment.
4. Use only following training by a healthcare professional.

Materials

1. Catheter – medical grade Thermoplastic Polyurethane (TPU) (Thermoformable).
2. Hydrophilic Comfort Coating – 2 layer coating with primer and topcoat Polyvinyl Pyrrolidone (PVP).
3. Packaging – completely recyclable.
4. 30 catheters in a box.
5. Supplied sterile, sterilised by Gamma Irradiation.

**Warning-precautions**

1. For single use only, do not reuse. Reuse can cause cross contamination.
2. This product is used for intermittent catheterisation only, and only on the recommendation of a health care professional.
3. Read the instructions prior to use.
4. Follow hygiene guidelines thoroughly as indicated in the Method of use section of this leaflet.
5. Do not use if the package or product has been damaged or contaminated.
6. Use immediately after opening and after use dispose of the catheter safely and responsibly.
7. Discontinue use if you experience any allergic reactions or problems when using the catheter and contact a healthcare professional for advice.

Storage conditions

Do not store at extreme temperatures and avoid direct sunlight. Store in a cool dry place.

Contra Indications

1. High intravesical pressure
2. Poor manual dexterity
3. Poor visual capability
4. Absence of a trained caregiver

Complications

1. Urinary tract infections (UTIs)
2. Urethral bleeding
3. Pain and/or soreness
4. Urethral stricture

Serious Incidents

Should a serious incident arise as a result of using the sterile Curan Advantage Lady PVC-free catheter, please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

Shelf life

24 Months.

Use by the expiry date noted on the product label.

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