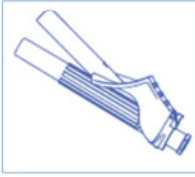
KD-FINE® SAFETY

WITH NEEDLE PROTECTION DEVICE

GB Hypodermic needle with needle protection device

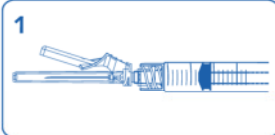
Brief description of the product: The safety mechanism of KD-FINE® SAFETY covers the needle after use. In the activated position, the safety shield guards against accidental needlestick injuries during handling of the used product and its disposal.

Materials used: Stainless steel, polypropylene (PP), silicone oil (lubricant), epoxy (adhesive)


Indications: Subcutaneous, intramuscular, intravenous and other soft tissue injections in conjunction with standard syringes or infusion/transfusion devices, blood collection

Contraindications: KD-FINE® SAFETY should not be used in patients with known hypersensitivity to any of the materials used.


Instructions for use:

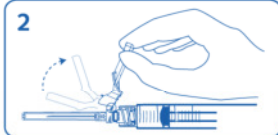
Carefully remove the safety needle from the package and securely attach it to a syringe with a push and clockwise twisting motion.

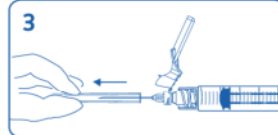Move the safety shield away from the needle and towards the syringe barrel. The safety shield will remain in the position you set.

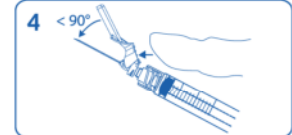Remove the needle cover carefully to avoid damaging the needle. Remove any air before use. Precisely adjust the medication by pushing the plunger until a medication drop emerges at the needle tip. Perform the intended procedure in accordance with currently accepted medical techniques.


Immediately after completion of the procedure position the safety shield in preparation for device activation: Using a one-handed technique, push the shield forward with thumb or finger so that the shield is less than 90° from the needle.

Activation can be made in two different ways (A or B):

When used with a Luer slip syringe, make sure the needle-syringe assembly is firm.


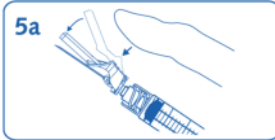
A


Center the thumb on the finger pad of the safety shield (5a). Activate the safety shield: Press down with a firm, quick motion until a distinct audible click is heard (6a).

NOTE: The activation force shall be applied vertically in the activation direction of the safety shield.

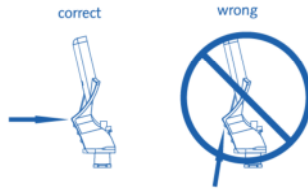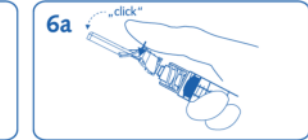
The device may detach when using Luer slip syringe if improper or wrong activation procedure is used.

B

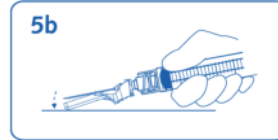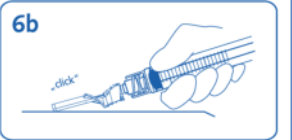
Position the safety shield in the direction of activation at the lowest possible angle to a flat surface (5b). Activate the safety shield: Press down with a firm, quick motion until a distinct audible click is heard (6b).

The audible click may be missed for small needle sizes. Visually confirm that the needle is fully engaged under the lock (7). *The location of the lock differs according to the needle size.

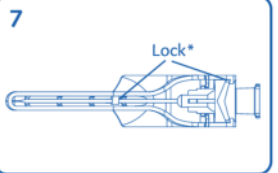NOTE: If the activation failed, dispose of the device immediately into an approved sharps container.

8. Dispose of the used product following federal and local regulations for sharps disposal.

Warnings:

- The use of this product is restricted to qualified healthcare professionals.
- Read instructions before use. KD-FINE® SAFETY should be used according to these instruction for use.
- Observe universal precautions on all patients.
- KD-FINE® SAFETY is guaranteed non-toxic, sterile and non-pyrogenic if the individual package has not been opened or damaged.
- Visually inspect and carefully check the device and packaging before use. Improper transport and handling may cause structural and/or functional damage to device or packaging.
- Do not use if the individual package is damaged, opened or contaminated.
- The product should be used immediately after opening of the package.
- If the needle is bent or damaged, do not try to straighten or use the product.
- Handle with care to avoid needlestick injuries. Keep hands behind the needle at all times during use and disposal.
- Do not replace the original needle cover after use.

- Activation of the safety mechanism may cause minimal splatter of any fluid remaining on the needle after injection. For maximum safety, activate the safety mechanism by using a one-handed technique away from self and others.
- Do not attempt to deactivate the safety mechanism by forcing the needle out of the safety shield.
- For single use on a single patient only.
- Re-use of single-use devices creates a potential risk for patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient.
- Discard after single use following federal and local regulations for sharps disposal.
- Do not clean or re-sterilize.
- KD-FINE® SAFETY must not be re-used.
- Store in a cool and dry place.
- Do not expose to heat or direct sunlight.

- Manufacturer
- Catalogue number
- Batch code
- Date of manufacture
- Use-by date

- Keep dry
- Keep away from sunlight
- Do not use if package is damaged
- Do not re-use

- Do not resterilize
- Consult instructions for use
- Caution
- Latex-free

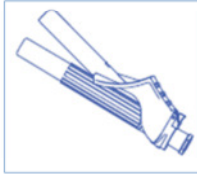
- Non-pyrogenic
- Sterilized using ethylene oxide
- CE 0123

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