- BE** Distribué par:
B. Braun Medical N.V./S.A.
1831 Diegem - Belgium
- DE** Vertrieb:
B. Braun Melsungen AG
34212 Melsungen - Germany
- ES** Distribuido por:
B. Braun Medical SA
08191 Rubí (Barcelona) - Spain
- FR** Distribué par:
B. BRAUN MEDICAL SAS
92210 Saint-Cloud- France
- GB** Distributed by:
B. Braun Medical Ltd.
Sheffield, S35 2PW - United Kingdom
- IT** Distribuito da:
B. Braun Milano S.p.A.
20161 Milano - Italy
- NL** Gedistribueerd door:
B. Braun Medical B.V.
5342 CW Oss - Netherlands
- AT** Vertrieb:
B. Braun Austria GmbH
2344 Maria Enzersdorf - Austria
- AU** Distributed by:
Omnigon Pty Ltd
Victoria 3039 - Australia
- BG** Дистрибутор:
"Б. Браун Медикал" ЕООД, бул.
София 1592 - Bulgaria
- CH** Distribué par:
B. Braun Medical AG
6203 Sempach - Switzerland
- CZ** Dováží:
B. Braun Medical, s.r.o.
148 00 Praha 4 - Czech Republic

- DK** Forhandles af:
B. Braun Medical N/S
2000 Frederiksberg - Denmark
- FI** Maahantuaja:
B. Braun Medical OY
00350 Helsinki - Finland
- HU** Forgalmazza:
B. Braun Trading Kft
1023 Budapest - Hungary
- JP** フレクシマ:
B.BRAUN AESCULAP JAPAN CO.,LTD
Tokyo - Japan 113-0033
- MY** Distributed by:
B. Braun Medical Industries Sdn Bhd
47800 Petaling Jaya, Selangor - Malaysia
- PL** Dystrybutor:
Aesculap-Chifa Sp. z o. o.
64-300 Nowy Tomyśl - Polska
- PT** Distribuido por:
B. Braun Medical Lda
2730-053 Barcarena - Portugal
- RO** Distribuira:
B. Braun Medical S.R.L.
Remetea Mare nr. 636 DN 6,
307350 Timiș, România
Tel. 0256 28 49 05
www.bbraun.ro
- RU** Дистрибутор:
Б.Браун Медикал
191040, Санкт-Петербург - Russia
- SE** Distribuerad av:
B. Braun Medical AB
182 11 Danderyd - Sweden
- SK** Distribútor da:
B. Braun Medical s.r.o.
83103 Bratislava - Slovakia

Flexima® Uro Silk / Softima® Uro Silk


- EN** Instructions for use
- DE** Gebrauchsanweisung
- ES** Instrucciones de uso
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- NL** Gebruiksaanwijzing
- BG** Указания за употреба
- CS** Návod k použití
- DA** Brugsvejledning
- FI** Käyttöohje
- HU** Használati utasítás
- JA** 使用方法
- PL** Instrukcja użycia
- PT** Instruções de utilização
- RO** Instrucțiuni de utilizare
- RU** Инструкция по применению
- SK** Návod na použitie
- SV** Brugsvejledning

Flexima® Uro Silk / Softima® Uro Silk

Flexima® Uro Silk / Softima® UroSilk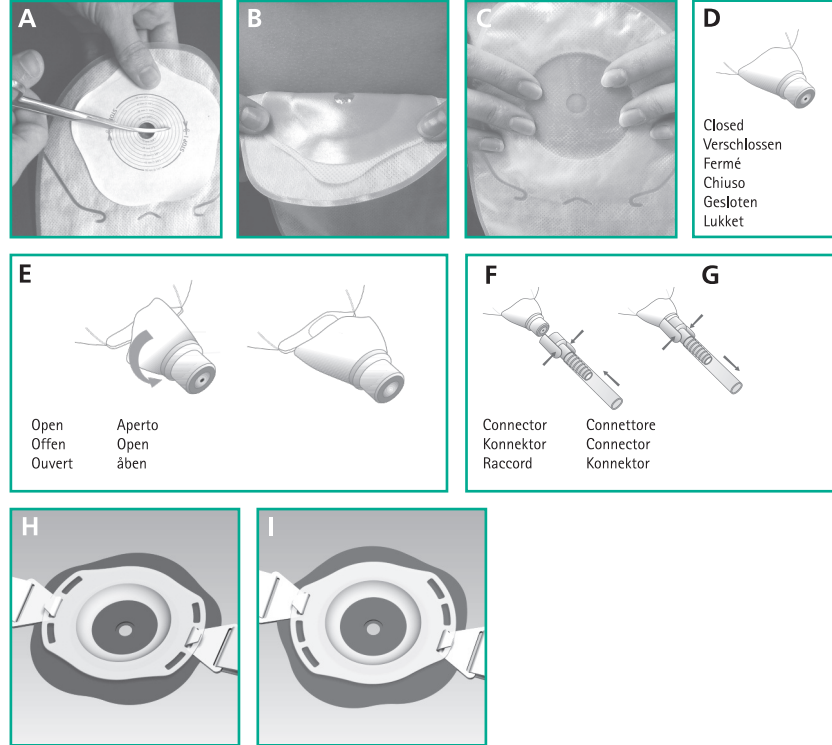

EN Instructions for Use

One-piece drainable pouch for collection of body effluents for any person having undergone surgery of the urostomy type.
2 types of collecting pouches *:

- Flexima® Uro Silk: total skin protector, flat or convex

1 Preparation of the skin:

- Wash your hands.
- Gently clean the area around the stoma with lukewarm water and carefully dab with soft tissues.
- Avoid using any disinfectant (ether, alcohol or other antiseptic...) as it damages the skin and prevents the proper adhesion of the pouch.

2 Fitting your appliance:

- Cut the skin protector so it fits the size of your stoma, taking good care not to perforate the film of the pouch (do not cut beyond the limit) (A).
- The pouch is delivered with the cap already closed, ready to be used (D).
- Separate the pouch walls by crumpling it slightly to make air penetrate.
- Peel the protective paper(s).
- Position the bottom of the flange opening around the base of the stoma (B).
- Press the bottom part of the skin protector smoothly onto the skin, then place and press onto the upper part.
- Smooth the entire surface of the skin protector, working from bottom to top, to ensure perfect adhesion (C).

3 Removing your appliance:

- Pull slightly on the top of the pouch and remove the pouch from the top to the bottom, while maintaining gentle pressure on the skin around the stoma with your other hand.

4 Urostomy pouches (emptying):

- With the pouch pointing downward, direct the drain cap into the toilet.
- Gently rotate the cap counterclockwise until you achieve the desired flow rate (E).
- To close the cap when the pouch is empty, return the cap to the original position by turning it clockwise.

5 Use of the connector for the night bag:

- Clip on the connector at the end of the closed cap by pushing it slightly (F). Open the cap turning it half a turn counterclockwise to allow urine to flow in the night bag. The connector can freely rotate to avoid tubing twisting.
- To un-clip, close the cap of the bag by turning it half a turn clockwise. Take the connector by the two wings and pull with caution (G).

The Flexima® urostomy bag can be connected to standard urine bags and to the Urimed® bag 2L.

CAUTION:

- Take good care to crumple the pouch slightly when fitting the appliance in order to separate the walls so that urine can flow freely!
- After the complete emptying of the pouch, pay attention to close the outlet completely by aligning it with its basis.
- If you wish to use a belt, choose two notches that are diametrically opposed (H, I).

* Availability may vary from one country to another.

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