UK/US/CA/AU

PRODUCT DESCRIPTION

Single use, hydrophilic urinary catheter with salt solution for activation.

MATERIAL

Hydrophilic coated urinary catheter. Polyolefin-based elastomer (POBE) with PVP coating. Sachet with NaCl and water.

INDICATIONS FOR USE

For short and long term bladder management with intermittent urinary catheterization.

PATIENT POPULATION

For adults, adolescents and children.

WARNINGS AND PRECAUTIONS

For single use only. Once used the surface coating will deteriorate and is no longer sterile.

Reuse may lead to discomfort, urethral damage or infection.

Do not use a product if the sterile packaging is broken or damaged.

Urinary catheterization therapy is associated with an increased risk of urethral bleeding, trauma and/or infection. LoFríc catheters are for prescription use only.

Follow instructions and advice from your healthcare professional.

Contact your prescriber if you experience difficulties.

ADVERSE REACTIONS

Common adverse reactions (>1/100) related to catheterization therapy includes urethral damage and urinary tract infection.

If unexpected discomfort, sign of trauma or infection occurs, discontinue use and consult your prescriber.

Any serious adverse reaction occurring when using the catheter should be reported to the manufacturer and your local health authority.

PACKAGE AND STORAGE

Store in their package in a dry place, at room temperature.

Use before expiry date on package.

UK: For more information please go to www.wellspect.co.uk, or call us at (0)1453 791763

US: For more information please visit www.wellspect.us, or call us toll free at 877-4-LOFRIC (456-3742).

AU: For more information please go to www.wellspect.com.au

CA: For more information please go to www.wellspect.com

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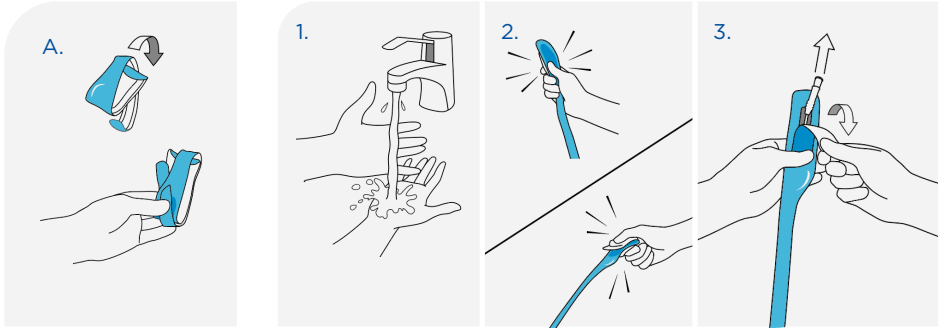
UK/US/CA/AU

A. Tips for storing before use. Foldable to pocket size. Use the slit to keep the catheter folded.

INSTRUCTIONS FOR USE

1. Wash your hands thoroughly with soap and water.

2. Press to release the salt solution and the catheter is ready to use.

3. Pull the tab down to open.

4. Optional: Use the adhesive tab on the reverse side to attach the product to a dry, clean surface.

5. Take out the catheter. Optional: Pull and adjust the Insertion Grip located on the funnel, to control insertion without having to touch the catheter tube.

6. Lift the penis towards the stomach to straighten the urethra. Slowly insert the catheter into the urethra. When urine begins to flow, insert the catheter slightly more to ensure both eyelets are inside the bladder.

7. Angle the penis down as urine begins to flow through the catheter.

8. When the urine flow stops, slowly withdraw the catheter a little bit. If urine starts to flow again, wait until it has stopped to ensure complete bladder emptying. Then remove the catheter completely.

9. Put the catheter back in the package and dispose appropriately (local regulations may vary).

Special instruction for Tiemann/Coudé urethral, with slightly curved tip. A special technique is required when using a Tiemann/Coudé. Speak to your healthcare professional for training and advice.

1. Note where the marker on the funnel is in relation to the curved catheter tip before inserting the catheter. It will guide you keeping the curved tip in the right direction during use.
2. Keep the curved tip upwards towards the stomach during insertion and throughout catheterization, including withdrawal. Or follow specific instructions given by your healthcare professional.

IMPORTANT INFORMATION Use only the salt solution included to wet the catheter.

Do not place the catheter on paper or cloth as these materials can absorb the wet coating.

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