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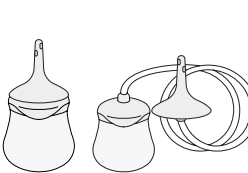
EN

Indications for using transanal/rectal irrigation

The Qufora IrriSedo MiniGo/Qufora IrriSedo MiniGo Flex is intended for adults and children 3 years and above to help manage faecal incontinence and/or constipation conditions.

For example:

- Passive faecal incontinence
- Post defaecation seepage
- Rectocele
- Incomplete evacuation
- Evacuation difficulties
- Low anterior resection syndrome

Within these groups some individuals may require the support of a carer to perform the procedure.

For the best results Qufora IrriSedo MiniGo/Qufora IrriSedo MiniGo Flex should be used daily or as advised by your healthcare professional.

Before starting transanal/rectal irrigation

Use only after assessment and training from a qualified healthcare professional. A digital rectal examination is mandatory prior to your first irrigation. You should be supervised by an experienced healthcare professional during your first irrigation.

Transanal/rectal irrigation should not be used under the following circumstances (absolute contraindications):

- Known anal or colorectal stenosis
- Colorectal cancer/pelvic malignancy pre surgical removal
- Acute inflammatory bowel disease
- Acute diverticulitis
- Within 3 months of anal or colorectal surgery
- Within 4 weeks of endoscopic polypectomy
- Ischaemic colitis

Since the list is not exhaustive a healthcare professional should always consider individual patient factors as well.

Use transanal/rectal irrigation only after careful discussion with relevant medical practitioners under the following circumstances (relative contraindications):

- Inflammatory bowel disease (e.g. Crohn's disease or ulcerative colitis)
- Active perianal sepsis (fistula or abscess, third or fourth degree haemorrhoids)
- Previous rectal or colonic surgery
- Diarrhoea of unknown aetiology
- Faecal impaction/rectal constipation
- Severe autonomic dysreflexia
- Severe diverticulosis or diverticular abscess
- Abdominal or pelvic irradiation
- Long term steroid therapy
- Anti-coagulant therapy
- Low blood sodium
- Previous severe pelvic surgery
- Colonic biopsy within the past 3 months
- Use of rectal medications for other diseases which may be diluted by irrigation
- Congestive cardiac failure
- Pregnancy or planned pregnancy or breastfeeding
- Children under 3 years of age

Warning

Seek medical attention if the following is experienced during or after transanal/rectal irrigation:

- Sustained or severe bleeding from the rectum
- Sustained or severe abdominal or back pain, with or without fever

Perforation of the bowel wall is extremely rare. However it is a serious and potentially fatal complication of transanal/rectal irrigation. Should this occur, immediate medical attention is required.

Please observe

Transanal/rectal irrigation can be associated with passing sensations of bowel discomfort, nausea, shivering, fatigue, sweating, headache and light bleeding from the rectum.

Please contact your healthcare professional for further advice.

Recommended usage time

The soft pump can be used 30 times (equal to 1 month usage time*).

Should discolouration develop within the recommended usage time, the soft pump should be replaced.

The cone/cone flex is for single-use only

If the cone/cone flex is reused, there is a risk of adverse effects to the product and the health of the patient, carers and third persons.

Liability

Qufora A/S accepts no liability for any injury or other loss that may arise if this product is used in any way contrary to the recommendations of Qufora A/S.

If any serious incident has occurred you should contact your healthcare professional who will liaise with Qufora A/S and the competent authority.

Please find further information about this product on www.qufora.com.

*when used for irrigation every day.

Distributed by:

Qufora Ltd,
Euro House,
Satellite Park,
Macmerry,
East Lothian,
EH33 1RW

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