Part no.	315-4289100-XXX
Product name	簡易使用指南/Diatesse XPER/NL EN FR DE/Asia Intermediates/
Spec	L297*W420mm/銅版紙100P/雙面/4C/3折(短邊對1折+長邊對2折)完成尺寸L148.5*W105mm
Color	Tsaiyi
Designer	 C55 M53 Y75 K4 (C50 M50 Y70 K0)  C47 M0 Y18 K0 (C50 M0 Y20 K0)  K85 (K80)

DIATESSE
XPER

Blood Glucose & KETONES Monitoring System

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Quick Start Guide

More information and instruction videos: www.diatesse.nl

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TaiDoc Technology Corporation
B1-7F, No.127, Wugong 2nd Rd., Wugu Dist.,
24888 New Taipei City, Taiwan

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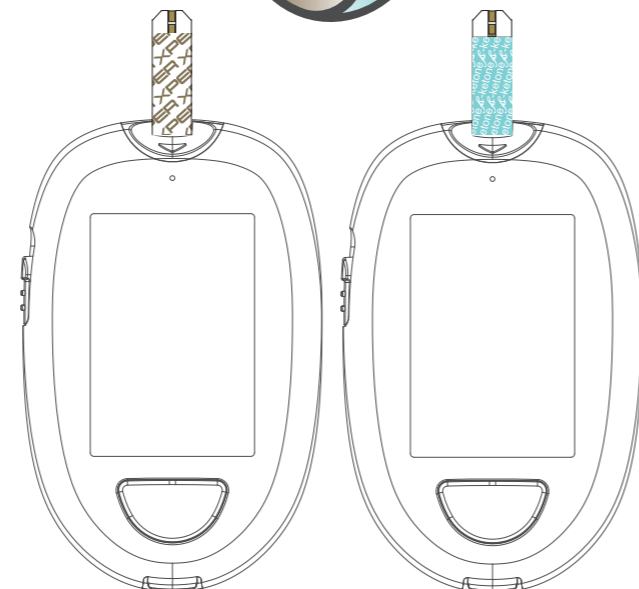
**IMPORTANT SAFETY INFORMATION
READ BEFORE USE**

1. Use the meter and separate components ONLY for the intended purpose as described in the enclosed user guides.
2. DO NOT use any accessories and/or components not described in the enclosed user guides.
3. Do not use the meter or separate components if damaged and/or not working properly.
4. This meter is NOT a medical treatment device. The measured data should only be interpreted as a reference. Always consult your doctor and/or diabetes nurse for accurate interpretation of the measured values.
5. Before using the meter, read all accompanying instructions and carefully practise the procedure. Perform all quality checks as described.
6. Keep the meter and separate components out of reach of young children. If swallowed, small components could create a choking hazard.
7. If using the meter in a very dry environment and particularly when synthetic materials are present (clothing, carpet, etc.), take care to avoid a static discharge, especially when touching the meter and test strips. If the meter and/or test strips are exposed to a static discharge they may produce incorrect measurement results.
8. Do not use the meter in the vicinity of strong electromagnetic radiation. This could have an adverse impact on accuracy.
9. Good maintenance and regular tests with the control fluid are important for the reliability and life of your meter. If you have concerns about the accuracy of your measurements, contact customer service or your supplier.

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