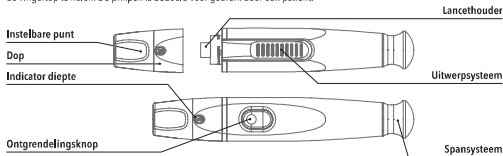
318-5011100-XXX Versie 3.0 2023/6

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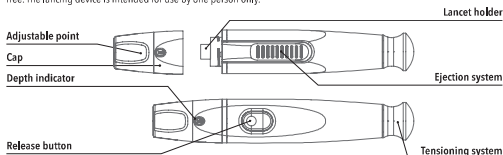
USER INSTRUCTIONS FOR THE LANCING DEVICE AND STERILE LANCET


The lancing device is used with sterile (universal) lancets to extract capillary blood from a fingertip easily and almost pain-free. The lancing device is intended for use by one person only.

WARNING


- ▶ Do not share the lancets or lancing device with others.
- ▶ Always use a new sterile lancet. Lancets are single-use only.
- ▶ Ensure that the lancets and lancing device are free of hand cream, grease or dirt.
- ▶ The used lancet could be a biologically hazardous object, so dispose of it in accordance with the applicable local regulations.

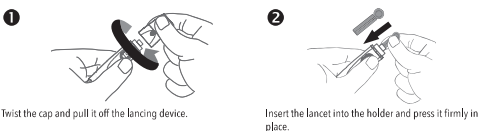Preparing the lancing device

Twist the cap and pull it off the lancing device.

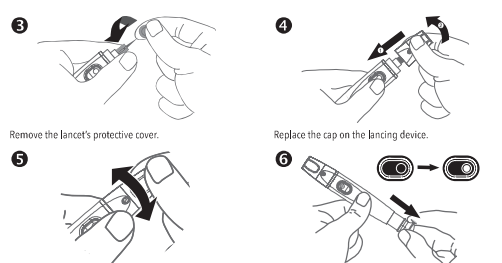Insert the lancet into the holder and press it firmly in place.

Remove the lancet's protective cover.

Replace the cap on the lancing device.

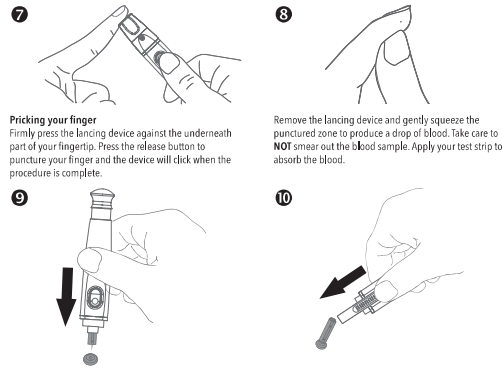
Place your finger on the penetration depth button. Select the penetration depth by turning the adjustable end, as illustrated.

Pull the tensioning handle out until it clicks into place. The release button will change colour when the device is ready.

Penetration depth:
1 - 3 soft to sensitive skin
4 - 6 normal skin
7 - 9 thick to calloused skin

NOTE:

- Before taking a blood sample:
- Wash your hands thoroughly.
 - Dry your hands well after washing them.
 - Select a good puncture point (preferably on the side of your finger).
 - Rub the selected spot for around 20 seconds before puncturing it.

Pricking your finger
Firmly press the lancing device against the underneath part of your fingertip. Press the release button to puncture your finger and the device will click when the procedure is complete.

Remove the lancing device and gently squeeze the punctured zone to produce a drop of blood. Take care to NOT smear out the blood sample. Apply your test strip to absorb the blood.

Unscrew the cap of the lancing device. Place the lancet's protective cover on a hard surface and carefully insert the lancet needle into it.

Slide the ejector forwards to eject the used lancet. Dispose of the lancet in accordance with the local regulations.

Cleaning and disinfecting the lancing device and cap

The disposable lancet is for single use only. The lancing device is reusable. Use a mild soap and water to wipe the lancing device clean. Do NOT rinse it in water. Disinfect the cap by placing it in a 70% alcohol solution for 10 minutes and then air-drying. Do NOT place the device in a dishwasher and do not use detergents.

Please note:

1. If the test result obtained from the puncture does not match how you feel, contact your doctor.
2. If the lancing device is operated by another person assisting the patient, then that person must clean and disinfect it before use. That person must also wear protective gloves during all steps while assisting in the test.
3. Always keep the lancing device, the lancet and all accessories out of reach of children.
4. Do not leave a lancet inserted in the lancing device.
5. Report any serious incidents involving this device to the manufacturer.

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TaiDoc Technology Corporation
MedNet EC-REP GmbH