*INSTRUCTIONS*

*ANLEITUNG  
INSTRUCTIONS  
HANDLEIDING  
INSTRUKTIONER  
INSTRUKTIONER  
BRUKSANVISNING  
OHJEET  
INSTRUCCIONES  
ISTRUZIONE*

*The Donning  
of a New Age™*

---

## INDEX

---

Pictures	4
English	6
Deutsch	9
Français	12
Nederlands	15
Dansk	18
Svenska	21
Norsk	24
Ranska	27
Español	30
Italiano	33

**ROLLING THE SOCK/STOCKING** *Rollen des strumpfes - Rouler le bas - Rollen van de kous - Pårulning af strømpen*  
*- Rullande av strumpa - Rulle strømpen - Sukkien rullaus - Enrollar la media - Srotolare la calza*

1A

1B

2

3A

3B

3C

4

5A

5B

PUTTING ON THE SOCK/STOCKING *Anziehen des strumpfes - Enfiler le bas - Aantrekken van de kous - Pãtrækning af strømpen - Pãtagning av strumpa - Ta pã strømpen - Sukan Pukeminen - Colocar la media - Indossare la calza*

6

7A

7B

8A

8B

9

10

11

REMOVING THE SOCK/STOCKING *Ausziehen des strumpfes - Enlever le bas - Uittrekken van de kous - Aftrækning af strømpen - Avtagande av strumpa - Ta av strømpen - Sukan riisuminen - Retirar la media - Sfilare la calza*

12

## DOFF N' DONNER

### **Usage:**

The Doff n' Donner can be used to put on (don) or remove (doff) almost all kinds of compression sock or stocking, regardless of the type, size or compression class. In doing so, the Doff n' Donner makes use of a hydraulic system.

### **Non-usage:**

Do not use this device if you suffer from hypersensitive skin or deep-vein thrombosis.

### **Instructions for use:**

Because the Doff n' Donner can be used with almost any kind of compression sock or stocking, there are also many ways in which to use it. Our website contains numerous instruction videos for various scenarios. You may also ask your compression sock/stocking specialist for help with your particular situation.

### **Warnings and precautions:**

- Do not heat this product in any way whatsoever
- Do not use this product if you suffer from deep-vein thrombosis
- Keep this product out of reach of children and pets
- Do not use this product on your head

- Remove sharp objects, such as jewelry, before use
- Consult your specialist before use
- For use by one person only, unless the Doff n' Donner is disinfected in the correct way

### **Cleaning and maintenance:**

The Doff n' Donner should be stored at room temperature, preferably in its original bag and box. The Doff n' Donner can be cleaned using normal (liquid) soap and water.

The Doff n' Donner can be disinfected with isopropyl alcohol. Special disinfection bags are available for this purpose.

The Doff n' Donner is filled with soapy water. This can easily be removed from clothing or floor coverings if the Doff n' Donner should leak.

---

## ROLLING THE SOCK/STOCKING

---

### PHOTO 1A+B

- Place the Cone on a suitable smooth and clean surface and press down the lever on the suction cap to form a firm seal

### PHOTO 2

- Place the sock/stocking over the Cone  
*Closed toe:* make sure the upper section is smooth  
*Open toe:* toe band should be placed below the bulge  
*Long sock/stocking:* the upper section should be smooth. The longer the sock/stocking, the more material will be at the base

### PHOTO 3A+B+C

- Place your fingers underneath the sock/stocking material using the indentations  
*N.B. when fitting a long sock/stocking, place your fingers further down under the material until they reach the indentations*
- Place your thumbs on the DnD
- Slowly pull the DnD up using your thumbs and fingers  
*N.B: do not fold the sock/stocking on the roller; allow the hydraulic system to do its work*

### PHOTO 4 (short socks/stockings only)

- Use both hands to roll the DnD up and off the Cone

### PHOTO 5A+B (long socks/stockings only)

- For long socks, hold the hem firmly above the Cone with one hand
- Then roll the DnD up and off the Cone with the other hand

---

## PUTTING ON THE SOCK/STOCKING

---

### PHOTO 6

When the DnD has been rolled off the Cone, the upper end is where the sock/stocking can still be adjusted. The sock/stocking toe is now located at the bottom end

### PHOTO 7A+B (closed-toe socks/stockings only)

- Insert your hand into the upper end of the DnD
- Then use your hand/fingers to make the sock/stocking toe nice and smooth

### PHOTO 8A+B (open-toe socks/stocking only)

- Insert your hand into the upper end of the DnD

- Then use your hand/fingers to adjust the toe band until it is roughly 2 cm (2 fingers) away from the edge of the DnD
- Adjust the hem to the correct position
- Repeat the process for the other sock/stocking

#### **PHOTO 9**

- Cross your legs
- Place the smooth sock/stocking toe or toe band against your toes
- Make sure that the sock/stocking heel and/or hem is facing down

#### **PHOTO 10**

- Use both hands to pull the DnD up over your ankle
- You may now let go of the DnD
- Make sure that the sock/stocking toe and heel are in the correct position.

#### *Please note:*

- *If the sock/stocking is positioned incorrectly, roll the DnD back down. The sock/stocking will roll back down for you to try once again. With an open-toe sock/stocking, try adjusting the distance to the toe band*
- *The flatter the foot, the easier it will be to roll the sock/stocking up over your ankle*

#### **PHOTO 11**

- Use both hands to roll up the sock/stocking; past the knee for a long sock/stocking
- Remove the DnD

---

## REMOVING THE SOCK/STOCKING

---

#### **PHOTO 12**

- Pull the DnD over your leg and up to the hem of the sock/stocking
- Take the hem of the sock/stocking and the DnD between your thumbs and fingers
- Roll the sock/stocking down to your ankle
- Cross your legs and use both hands to roll the DnD over your ankle and off your foot
- Remove the sock/stocking from the DnD, if necessary using the cone
- Repeat the process for the other sock/stocking

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