Cutimed® Siltec® L

Product description

Cutimed® Siltec® L dressings are an assortment of silicone coated foam dressings. Cutimed® Siltec® L is a sterile, absorbent polyurethane foam dressing for traumatic dressing changes that contains super-absorbent strips which absorb and lock wound exudate. The wound contact surface is a perforated silicone layer that allows adherence to the wound skin but not to the moist wound bed or to newly formed epithelial tissue. This minimises trauma and pain during dressing changes. The foam dressing and pain during dressing changes are designed to absorb and to lock away excess wound fluid inside the dressing, preventing it from coming into contact with the surrounding skin.

Cutimed® Siltec® L is a highly conformable dressing that offers protection of the wound site. Cutimed® Siltec® L leaves no residues in the wound. The dressing reliably retains wound exudate, even under compression. If necessary the dressing can be cut to size. Cutimed® Siltec® L is extra thin and soft for added wearing comfort.

Indications

The Cutimed® Siltec® L assortment is indicated for excising wounds such as venous and arterial ulcers, pressure ulcers, diabetic gangrenes, surgical incisions, skin grafts and donor sites, lacerations or abrasions. Cutimed® Siltec® L is recommended for wounds with low to moderate exudate levels.

Contraindications

Do not use on patients with a known sensitivity to any of the product components.

Warnings and precautions

Cutimed® Siltec® L is packaged for single use. Do not re-use or re-contaminate as there is a risk of transmission of body fluids or contaminated tissue to patients. Do not use if pouch is open or damaged. Discard if over or unused material. The wound should be inspected for signs of infection and treated according to clinical practice if required. Do not use with any irritating solutions such as hypochlorite or hydrogen peroxide. Remove the dressing prior radiation therapy.

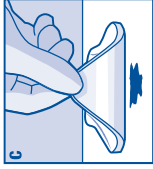
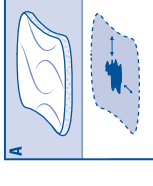
Directions for use

- Clean the wound according to good clinical practice.
- Make sure the surrounding skin is dry.
- Choose an appropriate dressing size that covers the wound and overlaps the wound edges. (A)
- Open the pouch and take out the dressing. Ensure sterility.
- Remove protective film (B) and apply with the adherent side towards the wound. (C)
- Avoid stretching the dressing.
- Apply additional dressing fixation.
- If necessary, Cutimed® Siltec® L can be used in combination with other dressings e.g. Cutimed® Sorbaact® (in case of increased wounds) or Cutimed® Der (for dry, necrotic wounds).
- Cutimed® Siltec® L should be applied to a selected wound in addition to antimicrobial therapy under medical supervision.
- Cutimed® Siltec® L can remain in place for up to seven days, depending on wound conditions and exudate levels. It is recommended to change the dressing every 24 hours initially, moving to less frequent changes as necessary.
- Carefully remove any additional dressing fixation and gently remove Cutimed® Siltec® L from the wound and discard.
- If required, cleanse the wound prior to application of a new dressing.

Storage

Cutimed® Siltec® L should be stored in dry conditions. Do not expose to direct sunlight.

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BSN medical GmbH