Vliwaktiv® Ag

Aktivkohle-Saugkompressie mit Silber
Absorbent Activated Charcoal Dressing with Silver
Compressa absorbente de carbón activo con plata

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en | Instructions for Use

Product Description:
Vliwaktiv® Ag Absorbent Activated Charcoal Dressing with Silver is an antimicrobial, highly absorbent, non-adherent wound dressing which contains an activated charcoal fabric impregnated with silver.

Vliwaktiv® Ag Absorbent Activated Charcoal Dressing with silver binds wound exudate with microorganisms and unpleasant odours. It thereby assists healing through an effective physical wound cleansing. The microorganisms in the wound exudate are removed from the wound by Vliwaktiv® Ag and bound and reduced through direct contact with the silver/activated charcoal fabric in the inside of the dressing. This reduces the bacterial growth inside the wound and prevents infection. The reduction of odour through the activated charcoal improves the quality of life and compliance of patients and medical staff.

Vliwaktiv® Ag Absorbent Activated Charcoal Dressing with silver is individually packed in a sterile peel pouch, and sterilized by gamma irradiation. The product is for single use only and must not be re-sterilized.

Indications:
Vliwaktiv® Ag Absorbent Activated Charcoal Dressing with silver is indicated for the treatment of:

- critically colonized wounds
- infected and/or offensive smelling wounds
- wounds with an increased risk of infection (e.g. from endogenous lowered resistance, anal fistulae, ruptured carcinomas, burns, etc.)

Vliwaktiv® Ag Absorbent Activated Charcoal Dressing with silver may be used for heavily exuding and superficial wounds, e.g. pressure sores, arterial ulcers, venous leg ulcers, diabetic ulcers, postoperative wound healing by second intention, exulcerating tumours.

Vliwaktiv® Ag Absorbent Activated Charcoal Dressing with silver may also be used as a secondary dressing in combination with packings (e.g. Suprasorb® A Calcium alginate rope, Vliwaktiv® Ag Activated Charcoal rope with silver).

Contraindications:
- Known sensitivity to the product or its components.
- Vliwaktiv® Ag should not be used in children less than 12 years old as the contained boron (in Borax) may have the potential to impair fertility in the future.
- The use of Vliwaktiv® Ag in pregnant and lactating women should be carefully considered by the clinician in charge as a case-by-case decision.

Application:
1. Wound Preparation: Before applying a new dressing, cleanse the wound in the usual way (e.g. with sterile saline solution, Ringer's solution or, if required, antiseptics) and gently dry the wound edges.
2. Application of the Wound Dressing: Apply Vliwaktiv® Ag Absorbent Activated Charcoal Dressing with silver with the dark activated charcoal side facing the wound. The size of the dressing depends on the wound size and should overlap the wound by at least 2-3 cm. Depending on the degree of exudation and the wound condition, the dressing may be combined with packings (e.g. Suprasorb® A Calcium alginate rope).
3. Dressing Change: The frequency of changing Vliwaktiv® Ag Absorbent Activated Charcoal Dressing with silver is determined by the clinician in charge dependent on the wound condition (e.g. signs of infection, degree of exudation).

Please note:
Vliwaktiv® Ag Absorbent Activated Charcoal Dressing with silver must not be cut or torn. The product is sterile unless the package is opened or damaged.

SVHC: Substances of Very High Concern ("substâncias extremamente preocupantes", según el Reglamento (CE) n.º 1907/2006 (REACH))

CMR: Carcinogenic Mutagenic Reprotoxic (cancerogénico, mutagénico, reprotoxíc)
Borax: Disodiuo Tetraborato Decahydrate, Na₂B₄O₇·10 H₂O, CAS no. 1303-96-4

SVHC: Substances of Very High Concern ("substâncias extremamente preocupantes", según el Reglamento (CE) n.º 1907/2006 (REACH))

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