# Comprilan® S

DE

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FR

## Short stretch bandages for support, relief and compression.

**Dressing instruction:**  
We recommend that bandaging be performed by a trained person.

**Indications:**  
Compression bandages for phlebological and lymphological indications to support muscle pump action and improve the haemodynamics of venous return; varicosis, chronic venous insufficiency including venous varicose ulcer and venous oedema, thrombophlebitis, phlebotrombosis, after vein operations (stripping), after sclerosing obliteration, primary and secondary lymphatic oedemas.

**Contraindications:**  
Advanced peripheral arterial occlusive disease, decompensated cardiac insufficiency, septic phlebitis, phlegmasia coerulea dolens, incompatibility to fabric.

**Cautions:**  
- Weeping dermatosis  
- Cutaneous infections  
- Open wounds must be covered with an appropriate dressing before compression bandaging is put on.  
- Severely compromised skin sensibility and impaired sensitivity of the limb  
- Advanced peripheral neuropathy  
- Conditions in which increased venous and lymphatic return is not desired.  
The attending physician decides whether compression in general can be applied to patients suffering from the following diseases: rheumatoid arthritis, complex regional pain syndrome, CRPS (M. Sudeck), malignant lymphedema, gangrene.

**Longitudinal elasticity:**  
Comprilan® S: approx. 90%  
Any serious incident that has occurred in relation to this device should be reported to BSN medical Inc. (BSN medical GmbH within the European Union) and the competent authority of your state.

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