3-WAY STOPCOCK, EXTENSION LINE

de Dreiwegehahn, verlängerungsleitung
fr Robinet 3 voies, prolongateur
sv 3-vägskran, förlängningssläng
fi 3-tiehana, jatkoletku
da 3-vejskan, forlængelsesslange
no 3-veiskran, forlengelsesslange
et 3-suunaline kraan, pikenduslini
lv 3 Ceļu savienotākrāns, pagarinātālinija
lt 3 krypčių čiaupas, išgnamoj linija
pl Trójnik, przewód przedłużający
ru 3-ходовой кран, удлинительная трубка
es Llave de 3 vías, línea de extensión
pt Torneira de 3 vias, linha de extensão
nl 3-wegkraan, verlenglijn

REF	Tube diameter (mm)	Tube length (cm)
10548-01	3 x 4,1	10
10549-01	3 x 4,1	30
10550-01	3 x 4,1	50
10551-01	3 x 4,1	60
10552-01	3 x 4,1	90
10553-01	3 x 4,1	100
10554-01	-	-
10571-02	3 x 4,1	10
10573-02	-	-
10574-02	-	-
10575-02	3 x 4,1	10
10624	3 x 4,1	100
10625	3 x 4,1	100
10699-01	1 x 3,1	15
10713	3 x 4,1	15

PUMP LINE ISO 8536-9-SPL-P
NEUTRAL CODE NO: OM20000

OneMed Group Oy
Metsäläntie 20, FI-00320 Helsinki, Finland

en

Materials Used:

Polyolefin alloy (PVC Free), High-density polyethylene (HDPE), Acrylonitril Butadiene Styrene (ABS), Polycarbonates (PC), Adhesive -Acrylate.

Intended Purpose:

IV-Line stopcocks are single use component contains valve to regulate the direction of liquid flow or/and as a part of extension of infusion & perfusion device.

Intended Users are Healthcare professionals. There is no restriction to a specific patient population.

IV-line stopcocks are sterile and intended for single use. IV-line stopcocks principle of operation is to regulate the direction of liquid flow and to conduct intravenous fluids as an extension of the infusion or perfusion device, by enabling connections to two sources of infusion and an infusion device with vascular access

Instructions for Use:

- Safe for use with pressure infusion equipment.
- Ensure aseptic technique is followed at all times.
- Visually inspect the product for any imperfections or surface deterioration before use.
- For safety reasons, tighten the caps and all disconnectable fittings before use.
- Connect a pre-filled syringe or IV administration set primed with sterile fluid to one of the female ports to prime.
- While priming, turn the tap using the directional arrows to desired port position and flush the fluid through the entire stopcock, repeat for all female ports and continue flushing until all air is completely removed.

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Warning:

- The use of this product is restricted to a qualified doctor or nurse.
- Read instructions before use and the device should be used according to these instruction for use.

The device is guaranteed sterile and pyrogen-free if the individual package has not been opened or damaged.

Do not use if package is damaged or if protective caps are loose or missing.

Do not over-tighten a luer beyond the friction fit as this may damage both the luer and another connecting device.

Do not use any instruments for clamping the tubing or luer tightening.

Re-use may lead to infection or other illness/injury.

Do not leave open ports/hubs exposed. Replace sterile end caps, as necessary.

Indication:

Used for clinical venous infusion of I.V. fluids, parenteral

nutrition, and administration of other drugs.

Contraindication:

Not to be used in patients with known hypersensitivity to any of the materials used.

Disposal:

According to local regulations.

Notice:

Serious incident that has occurred in relation to the device must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

These instructions for use were drawn up in English and have been translated into other languages. In the event of inconsistencies of interpretations, the English text shall be determinative.

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