Školská 668
093 02 Vranov nad Topľou
tel./fax: 057/442 12 86, 442 39 29, 446 41 67
e-mail: printon@printon.sk
web: www.printon.sk

CUSTOMER: Tytex Slovakia s.r.o.

TYPE: leaflet
NAME: Corsinel StomaSafe Classic Ostomy Support Belt 3pcs
P-nos: P2187.05-06-2021
DATE of PROC: 15.6.2021 TECH. CHECK: JP

DIMENSIONS: unfolded: 370×305
 folded: 185×232


OTHERS:
varnishing: NO
partial varnish: NO
cutting: YES
creasing: YES
folding: NO
gluing: NO

COLOURS: paper colour: WHITE
 separation colours: C M Y K
 direct colours: NO

Authorized signature:.....Date:.....

reverse side folding line

Please keep this instruction for further reference

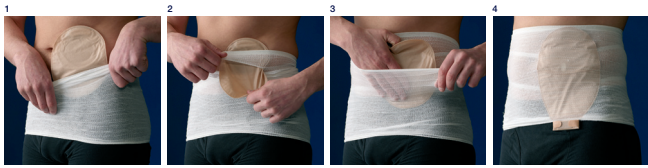EN
Indication
Medical device for fixation and support of an ostomy pouch.
Can be used for fecostomy, Colostomy and Urostomy.

Features
StomaSafe Classic is double layered with a hole for the ostomy pouch in the inner layer, which prevents contact between skin and pouch. The outer layer retains the ostomy pouch safely, preventing the pouch from moving.

How to measure yourself
Measure yourself in standing position and at the widest abdominal circumference covered by the StomaSafe.

NOTE: Sizes overlap; if abdominal measurement lies between two sizes, choose the smaller size if you prefer a more tight fitting StomaSafe, and the larger size if you prefer it less tight. Ensure that the StomaSafe feels comfortable to wear when sitting and standing.

Instructions for Use

1. Pull up the StomaSafe Classic (like a piece of underwear), with the large opening between the layers facing upwards.
2. Pull the pouch through one of the two holes in the inner layer. Choose the hole that fits your pouch/bag best and provides you with the best comfort and support.
3. Place the ostomy pouch in the inner pocket.
4. When using a drainable pouch, pull the drainage valve through the lower garment opening. Choose the opening that fits the placement of your drainage valve best.

Precision
If too small/light fitting, the product may influence the stoma drainage. If any questions regarding size, usage, indication, discomfort or problems occur, please consult your doctor or your nurse.

Care Handling
Wash with similar colours in a washing bag. Use of chlorine-based bleach or chlorine-based disinfectant will decrease durability and function of the product.

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