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### Qufora<sup>®</sup> Twostep Hydrophilic Catheter

#### FEATURES

Qufora<sup>®</sup> Twostep is a Hydrophilic Urinary Catheter. It is intended for intermittent drainage of the urinary bladder. The catheter consists of phthalate free PVC coated with cross-linked PVP (Polyvinylpyrrolidone). The hydrophilic surface of the catheter binds water and creates a highly slippery and low friction surface that eases insertion and withdrawal of the catheter.

The catheter connector has the international colour code as follows:

Size (CH)	06	08	10	12	14	16	18
Colour	Light green	Blue	Black	White	Dark green	Orange	Red

The catheter is not recommended for children.

#### USAGE

- Emptying of the urinary bladder, in the hospital or at home
- Intermittent catheterisation

#### STERILISATION

Sterilisation with Ethylene Oxide. Do not resterilise.

#### INSTRUCTION OF USE

- Wash hands before catheterisation.
- Peel the package at the connector end.
- Fill the package with water.
- Optionally attach the package onto any dry surface by using the self adhesive tape.
- The catheter must be soaked in water for at least 30 seconds before use.
- When the low friction is activated the water can be discharged from the package.
- The catheter is now ready to use.
- The package can be used as a non touch grip/slide to ease handling of the slippery surface without touching it during catheterisation.

- Insert the catheter into the urethra.
- Continue to insert the catheter into the urethra until urine starts to flow.
- Hold the catheter in place until the urine stops flowing.
- Remove the catheter by pulling it out slowly.

#### INGREDIENTS

Catheter body: Medical Grade PVC (phthalate free)  
 Connector: Medical Grade PVC (phthalate free)  
 Surface Coating: PVP K90  
 Packaging: Medical packaging paper suitable for EO sterilisation  
 Film : PP + LDPE-film  
 Box: Carton box

#### WARNINGS

The catheter is for single use only. The catheter must be disposed after usage. Do not use catheter if the package is open or has been damaged.

#### STORAGE

Store at room temperature (5-30° C). Store clean and dry away from heat and light. Shelf life 3 (three) years.

#### PACKAGING

Quantity per box: 30 pcs

**Manufactured for**  
 MBH International A/S  
 Gydevang 28-30  
 DK-3450 Allerød  
 Denmark

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MBH-International A/S