4-7-19

CF

Onestep

Instructions for Use

Onestep Hydrophilic Catheter for single-use

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02.08.19

100%
phthalate
free

Manufactured for
MBH-International A/S
Gydevang 28-30
DK-3450 Allerød
Denmark

Phone: + 45 8830 0592
Fax: + 45 8830 0593

Istem
MEDİKAL TIBBİ CHAZ VE SANAYİ LTD.ŞTİ.
Anadoluhisari Mah. 28 Ekim Cad. No:39
Kalkıcı/Sincan/Ankara, Turkey
A0387-01 28.05.2019

2292

Instructions for Use

Onestep

Hydrophilic Catheter for single-use

Intermittent self-catheterisation is a common procedure; however, you should always seek guidance from your healthcare professional and carefully read the instructions before use.

FEATURES

Onestep is delivered sterile with an activated hydrophilic surface. It can be inserted into the urethra right after its removal from the packaging. Onestep is made of phthalate-free PVC and is coated with cross-linked PVP (polyvinylpyrrolidone). Onestep is latex free. The catheter is stored in its own sterile water.

PRECAUTIONS

Catheterisation should be completed as soon as possible after the packaging is opened and the catheter should be removed once the bladder is empty.

In the case of potential urinary infection with symptoms such as fever, discomfort on emptying bladder, frequent need to empty bladder or blood in urine, or in the event of a serious side effect or side effects not indicated in the guideline (catheter break, allergy, risk of bleeding, fluid inside the device is cloudy and smelly) please contact your healthcare professional for advice.

USAGE

- Emptying of the urinary bladder, in the hospital or at home
- Intermittent catheterisation

STERILISATION

- Sterilisation with Ethylene Oxide.
- Do not resterilise.

INSTRUCTIONS OF USE

1. Wash hands before catheterisation.
2. Attach the catheter package to a smooth surface using the sticky dot.
3. Peel the package open and the catheter can be removed and used immediately. Please take care not to drop or contaminate the catheter.
4. Preparation for insertion of the catheter:

4.a. Female:
With one hand, part your labia to expose the urethral orifice.

4b. Male:

Straighten out your urethra by lifting your penis with one hand. Avoid squeezing the penis, as this may make insertion of the catheter more difficult.

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5. Insertion of the catheter

5a. Female:

Without touching the clear catheter, gently insert the catheter into the urethra - approximately 2 cm at a time. If the catheter has a non-touch slider, this can be used for insertion.

5b. Male:

Without touching the clear catheter, use the non-touch slider to gently insert the catheter into the urethra approximately 2 cm at a time.

Just before the catheter reaches the bladder you may feel a slight resistance due to the bladder sphincter.

6. Continue to insert the catheter into the urethra until urine starts to flow.

Note: The catheter can be connected to a urine drainage bag if there is no toilet nearby.

7. When the urine stops flowing, ease out the catheter 2 to 3 cm. If the urine flow restarts, wait a few seconds before easing out the catheter another 1 to 2 cm.

8. Once the bladder is completely empty, remove the catheter slowly and dispose of it safely.

POSSIBLE COMPLICATIONS

- Urinary tract infection (high probability)
- Bleeding due to trauma and infection (low probability)
- Epididymitis, urethritis, urethral stricture, kidney- and bladder stones (rare probability)

POSSIBLE SIDE EFFECTS

- Urinary tract infection, sepsis
- Hematuria, skin irritation, burning, pain sensation
- Urethral bleeding including urethral trauma and urethral stricture underlying potential bladder dysfunction
- Epididymoorkitis, epididymitis and pyelonefritis

WARNINGS

- The catheter is for single use only.
- The catheter must be disposed after usage.
- Do not use catheter if the package is open or has been damaged.

PACKAGING

Quantity per box: 30 pcs

The manufacturer accepts no liability or loss that may arise if this product is not used entirely according to the manufacturer's recommendation.

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MEDİKAL TIBBİ CİHAZ VE SANAYİ LTD. ŞTİ.