# DANSAC SKIN PROTECTOR WIPE

REF 081-30

- da • Hudbeskyttelse
- de • Hautschutz-Tücher
- en • Skin protector
- fr • Protecteur cutané
- it • Skin Protector
- pl • Ochrona skóry
- sl • Sredstvo za zaščito kože

dansac

[www.dansac.com/contact](http://www.dansac.com/contact)

[www.dansac.com/symbolglossary](http://www.dansac.com/symbolglossary)

912920

912920-520 ©2020 Dansac A/S

## da • Hudbeskyttelsesservietter

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## en • Skin protector wipes

**Contents:** Skin protector wipes.

**Product Description:** Silicone-based film designed to protect skin against stoma output and pouch adhesives. Alcohol free – no sting.

**Indications and Intended Use:** This product is an accessory to an ostomy skin barrier that provides a protective layer between skin and bodily wastes and fluids when used with ostomy products.

**Ingredients:** Hexamethyldisiloxane, Trimethylsiloxysilicate, Cyclopentasiloxane

**WARNINGS** Reuse of a single use product may create potential risk to the user such as poor adhesion.

**EXTREMELY FLAMMABLE:** Keep away from sources of ignition – No Smoking. Keep out of the reach of children. Use only as directed.

**Precautions** For external use only. To be used on intact skin only. Avoid contact with eyes.

### Notes:

If the protective wipe is used in an area with skin folds or other skin-to-skin contact, make sure the skin contact areas are separated to allow the film to thoroughly dry before returning to normal position. Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see [www.dansac.com/authority](http://www.dansac.com/authority) or contact local distributor.

### INSTRUCTIONS FOR USE

Skin should be clean and dry prior to application.

- 1 Apply a thin uniform coating to the skin area to be protected.
- 2 Let dry.
- 3 Apply ostomy pouching system.

Please follow all local disposal regulations.

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