INSTRUCTIONS FOR USE

StomiFix – STOMA BINDER

art no 50217

Size	Hip Circumference	
XSmall - Small	65 - 90 cm	
Small - Medium	85 - 110 cm	
Medium - Large	105 - 125 cm	
Large - X Large	120 - 140 cm	

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EN 1. Put on the tubular shaped stoma binder just as you would with regular underwear. Make sure the edge is facing upwards towards the waist, and that the pouch hole is placed over the ostomy pouch. Pull the bag out through the hole.

2. Fold down the upper part of the binder to cover the bag. Secure the pouch in the folded edge at the bottom of the binder. Wear your regular underwear underneath or on top of the StomiFix.

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EN In order to determine the correct size, measure the hip circumference (at its widest point) in centimeters.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of stoma binders. For best results, follow instructions for use.

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EN Machine wash in 60°C (140°F). Color lasts longer if washed in 40°C (104°F). Use non-bleaching detergent. Do not bleach or use softener.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent –or NordiCare: +46 42-35 22 20 / DK 7070-1907, info@nordicare.se.

**NordiCare**

 **NordiCare Ortopedi & Rehab**
Solrosvägen 1, SE-263 62 Viken, Sweden.
Tel +46 42 35 22 20, Fax +46 42 35 22 21
info@nordicare.se, www.nordicare.se

2020-04-08

 This product meets the requirements of Medical Devices Class 1, non sterile.

Available colours: