90-1040/4 (April 2021)

(en)

**Description of device & contents**  
Trio Siltac® Silicone ostomy seals  
Trio Silvex® Silicone convex seals

| Trio Siltac® Flat Ostomy Seal |           |
|-------------------------------|-----------|
| N°                            | Size (mm) |
| 1                             | 20-28     |
| 2                             | 28-35     |
| 3                             | 35-44     |

| Trio Silvex® Convex Ostomy Seal |           |
|---------------------------------|-----------|
| N°                              | Size (mm) |
| 1                               | 20-30     |
| 2                               | 30-40     |

**INTENDED USE:** To provide a seal between the stoma, peristomal skin and the ostomy pouching system.  
• Flat seal for use with protruding stoma.  
• Convex seal for use where mild convexity is required.

**INSTRUCTIONS FOR USE:** To ensure a secure seal is achieved, it is important to select the correct size of stoma. Should leakage occur you may select the next size down to achieve a tighter seal.

**APPLICATION**

1. Clean area with water and ensure that the skin is dry. Attention: The seal will not stick to wet skin.  
2. Rubbing by gently rubbing the edge of the seal toward the center; then using thumb and forefinger carefully but firmly pull from the packet. The seal will normally be adhered to the pocket.  
3. Stretch Trio Siltac® or Trio Silvex® over the stoma and allow it to retract. Once the seal has retracted around the stoma, secure to the body by gently pressing on the seal until secure and comfortable.  
4. Place your ostomy pouching system directly onto the surface of the seal and hold in place with gentle pressure until secure.

**REMOVAL**

- i. Gently peel the pouching system from the body.  
ii. Remove the seal from the skin surface. The seal may peel off with your pouching system.
- Caution:** Should peristomal skin irritation occur, discontinue use and consult a healthcare professional.

**SERIOUS INCIDENT REPORTING:** Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**SAFE DISPOSAL INSTRUCTIONS:** Do not dispose in the toilet. Dispose of product according to national or local laws.

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Trio Healthcare Ltd