90.1043/7 (April 2021)

(en)

Description of device & contents

Trio Silken® gel

INTENDED USE: As a filler / sealant and protective skin barrier for use on exposed skin around stoma, wound and fistula sites. A filler for skin folds and uneven skin surfaces.

INSTRUCTIONS FOR USE:

- 1 To open the tube, Unscrew the cap. Puncture the tube closure with the reverse side of the cap.
- 2 Ensure the skin is clean and dry.
- 3 Squeeze the silicone gel from the base of the tube, to fill depressions and skin contours. Do not spread.
- 4 Apply the ostomy wafer or pouching system as normal while the Trio Silken® gel is still wet.
- 5 Trio Silken® gel will set on the skin or underneath the appliance during wear time.

REMOVAL:

1. Peel the Trio Silken® gel from the skin surface. The gel may peel off with the appliance.

IMPORTANT NOTE:

Trio Silken® gel is not a glue.

CAUTION:

For external use only.
Do not apply into an open wound or skin ulcer.
Avoid contact with eyes.
Should skin irritation occur, discontinue use and consult a healthcare professional.

SERIOUS INCIDENT REPORTING

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SAFE DISPOSAL INSTRUCTIONS

Do not dispose in the toilet. Dispose of product, according to national or local laws.

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