

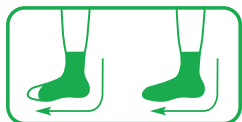
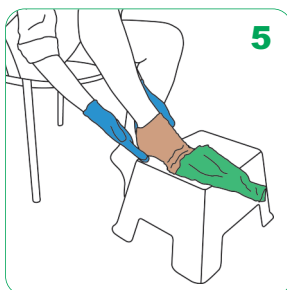
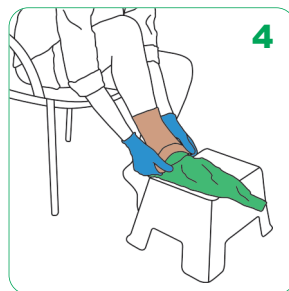
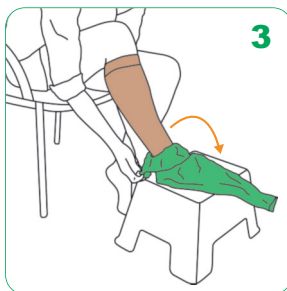
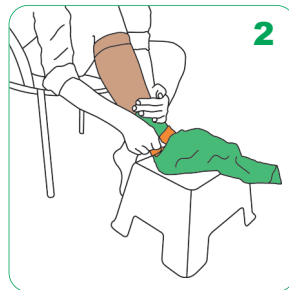
ARION Easy-Off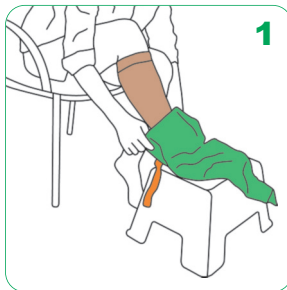

EN	Doffing aid for medical compression stockings	Instructions for use	2
DE	Ausziehhilfe für medizinische Kompressionsstrümpfe	Gebrauchsanleitung	4
NL	Uittrekhulp voor medische compressiekousen	Gebruiksaanwijzing	6
DA	Hjælp til aftagning af kompressionsstrømper	Brugsanvisning	8
FR	Aide au retrait de bas de compression médicale	Mode d'emploi	10
NO	Avtakingshjelp for medisinske kompresjonsstrømper	Bruksanvisning	12
SV	Avklädningshjälpmedel för medicinska kompressionsstrumpor	Bruksanvisning	14
FI	Riisumisen apuväline lääkinällisille kompressiosukille	Käyttöohjeet	16
ES	Descalzador de medias de compresión médica	Instrucciones de uso	18
PT	Auxiliar para descalçar meias de compressão médica	Instruções de utilização	20
IT	Dispositivo per togliere le calze compressive medicali	Istruzioni per l'uso	22
JP	医療用弾性ストッキングの脱衣補助具	使用説明書	24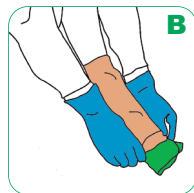

Arion Easy-Off

Doffing aid for medical compression stockings

Important information before you start using the product:

- A.** If you have difficulty to reach your toes, please place your foot on a low chair to avoid bending too much.
- B.** Wear special non-slip gloves when using the product. This gives you maximum grip on the stocking and reduces the physical strength needed.

Doffing instructions:

1. Slide the Easy-Off over the stocking
2. Fasten the Velcro strap firmly just above the ankle
3. Fold the spare material down over the Velcro strap as far as possible
4. Fold the stocking over the Easy-Off as far as possible
5. With small movements, starting at the heel, slide the stocking and the Easy-Off over the foot

Off

Watch our **video instructions** on uqr.to/OffEN

SIZING

REF	Size	Shoe size EU	Shoe size UK		Shoe size US	
			women	men	women	men
E01510	Small	34 - 37	1 ¹ / ₂ - 4 ¹ / ₂	2 ¹ / ₂ - 4	4 - 6 ¹ / ₂	3 - 5
E01511	Medium	38 - 41	5 - 7	4 ¹ / ₂ - 7 ¹ / ₂	7 - 9	5 ¹ / ₂ - 7 ¹ / ₂
E01512	Large	42 - 45	7 ¹ / ₂ - 10	8 - 11	9 ¹ / ₂ - 11 ¹ / ₂	8 - 10

Watch our **instruction video** here: uqr.to/OffEN or scan this QR code:

More information on www.slide-solutions.com

Easy-Off from Arion Slide Solutions minimizes friction between compression garment and skin thanks to the product's unique sliding properties provided by the special material it is made of and by its coating. The friction reduction is particularly helpful to slide the garment over the heel area, which is the broadest part of the foot.

EASY, SAFE AND SIMPLE SOLUTION

- Reduces physical strain for both compression wearers and caregivers
- May contribute to a longer life for the garment and to maintaining the original compression longer by preventing damage from fingers or nails and avoiding overstretching and distortion
- Promotes independence of compression wearers and supports therapy compliance

COMPOSITION

- Product: Polyamide 92%, Polyurethane 3%, Silicone 2%, Polyester 3%
- Package: Low-density polyethylene (LDPE)

CONTRAINDICATIONS

- Do not use the product if you are allergic to one of its component (see composition).
- Open wounds should be covered with an appropriate dressing before using the product.

! WARNINGS

- **Suffocation hazard:** Keep the product away from babies, children, and pets.
- **Slipping hazard:** Do not step on the product. Highly slippery material.
- **For external use only /** Intended for skin contact. Seek medical attention if irritation develops.

IMPORTANT

- Please **read the instructions for use carefully**. If you have any questions, please contact your reseller.
- Please **keep the package and the instructions for use during the time of use of the product**.
- All **serious incidents** that occur when using the product **should be reported** to the manufacturer and the competent supervisory authority.
- When using the product it is important to **use the right product size** for the user. Using an incorrect size will prevent the product from working properly, and it may become damaged or wear out too quickly. The correct size is determined by your shoe size.
- **Use the product on a single person**. If the product needs to be used on more than one person, disinfect the product before with a 70% ethanol solution.

STORAGE

Keep away from direct sunlight or other heat sources.

CARE INSTRUCTIONS

- Wash by hand, at max. 30°C/86°F.
- Do not bleach.
- Do not tumble dry. Dry the product only with a towel.
- Do not iron.
- Do not dry clean.
- Disinfection possible with a 70% ethanol solution.







GUARANTEE

This product is guaranteed for a period of six months from the date of purchase against defects resulting from faulty material or construction. Within the guarantee period, the product will be exchanged free of charge in the case of manufacturing and/or material defects. Any exchange under the guarantee will only be carried out if it can be demonstrated by means of a purchase receipt that the date on which the complaint was lodged falls within the guarantee period. The guarantee does not apply if the defect is due to improper use of the product or use on several patients. To use this service you need to contact your supplier. You are asked to return the product washed and to follow the care instructions.

RECOMMENDATION FOR USE

The manufacturer recommends replacing the product six months after the first use if the product is used regularly.

You can write the date of first use on the back of this document.

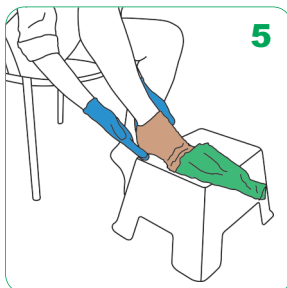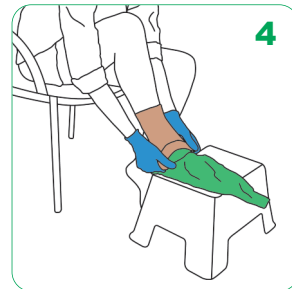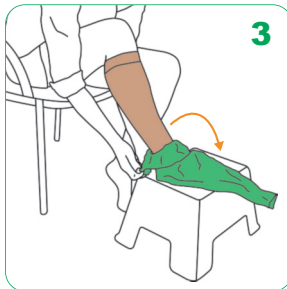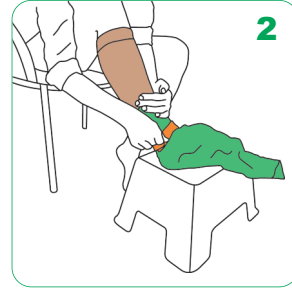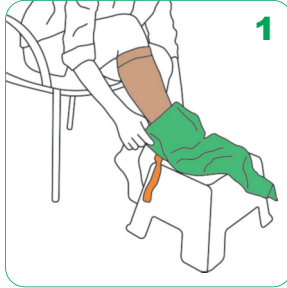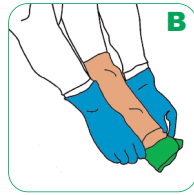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Arion Slide Solutions B.V.