DOFF N' DONNER SET bestående aus:

- A: DND
- B: CONE

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EN: CAUTION: If used frequently for a long period of time, the DND may become brittle and may tear out in exceptional cases.

We therefore recommend storing the DND in a safe place. We therefore recommend that the DND be stored protected from light and heat. When used and stored correctly, we guarantee perfect function for a period of 6 months from the date of purchase.

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EN: After use we recommend that the DND be cleaned and / or disinfected.

For private use it is recommended that the DND be cleaned in a commercially available plastic bag filled with soapy water. For institutional use (e.g. hospitals, doctor's surgeries, medical suppliers etc.), the DND must be disinfected or sterilized.

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EN:

The DOFF N' DONNER (DND) is a very soft device that allows you to quickly and painlessly convey medical compression stockings over an arm or a leg. The DND is suitable both for self-application and assisted application. The CONE simplifies the rolling of a compression stocking onto the DND. With the aid of the CONE, the medical compression stocking can also be easily unrolled from the DND. By virtue of the integrated suction cup, the CONE can be attached well to smooth surfaces.

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EN: Clamp the CONE to a smooth surface (press the lever down, CONE adheres firmly).

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EN: Place the compression stocking with the finished top edge (soft top or grip top) over the CONE. The inside of the stocking rests on the CONE.

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EN: Place the DND over the CONE and roll it downward. Pull the finished edge of the compression stocking up over the DND.

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EN: Calf stocking: Roll the DND upward until the compression stocking is rolled completely onto the DND.

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EN: Thigh stocking: Pull the finished edge of the compression stocking beyond the tip of the CONE, so that the stocking is wrinkle-free. Grasp the DND and pull it upward until the compression stocking is rolled completely onto the DND.

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EN: Stretch the toe seam, so that it is clearly visible from the outside. Hold the DND in such a way that the heel of the compression stocking is visible from the outside and is facing the floor.

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EN: Roll the DND over the heel up onto the leg. **Open toe:** Place the finished cuff as shown by the arrow.

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