

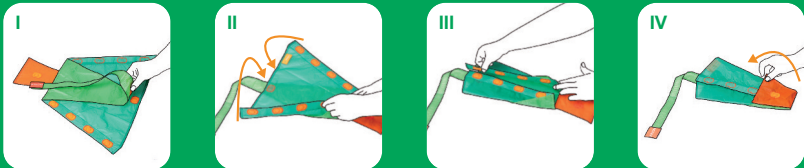
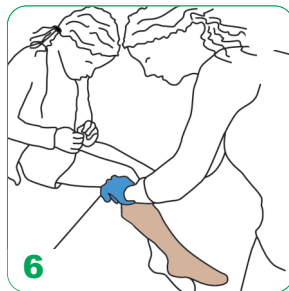
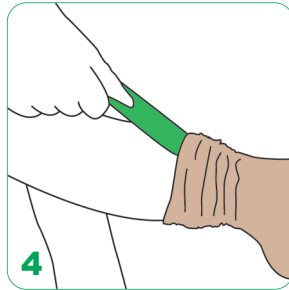
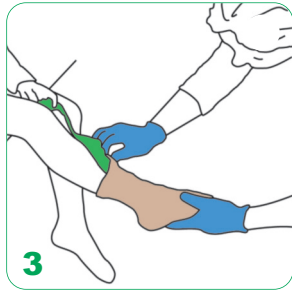
# ARION Magnide® Kids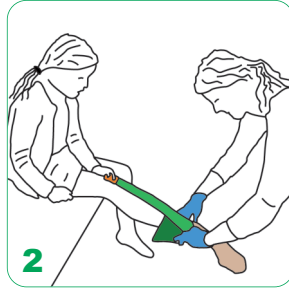

<b>EN</b>	Donning aid for medical compression stockings with a closed toe. For children	<b>Instructions for use</b>	<b>2</b>
<b>DE</b>	Anziehhilfe für medizinische Kompressionsstrümpfe mit geschlossenerer Fußspitze. Für Kinder	<b>Gebrauchsanleitung</b>	<b>4</b>
<b>NL</b>	Aantrekhelp voor medische compressiekousen met gesloten teenstuk. Voor kinderen	<b>Gebruiksaanwijzing</b>	<b>6</b>
<b>DA</b>	Hjælp til påtagning af kompressionsstrømper med lukkede tæer. Til børn	<b>Brugsanvisning</b>	<b>8</b>
<b>FR</b>	Aide à l'enfilage de bas de compression médicale à pointe fermée. Pour enfants	<b>Mode d'emploi</b>	<b>10</b>
<b>ES</b>	Calzador de medias de compresión médica con puntera cerrada. Para niños	<b>Instrucciones de uso</b>	<b>12</b>
<b>IT</b>	Dispositivo per mettere le calze compressive medicali con punta chiusa. Per bambini	<b>Istruzioni per l'uso</b>	<b>14</b>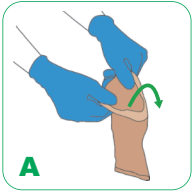

### Arion Magnide® Kids

Donning aid for medical compression stockings with a closed toe. For children.

#### Important information before you start using the product:

- A.** Useful tips:
- Wear special non-slip gloves when using the product. This gives you maximum grip on the stocking and reduces the physical strength needed.
  - In case the compression garment has a silicon border, make sure to turn it outwards.

#### Donning instructions:

1. Let the child sit in front of you, preferably on a higher bed or table
2. Place the Magnide® Kids across the foot of the child (ensure that the orange flap near the toes is well folded back, so that the magnet makes contact)
3. Pull the stocking over the Magnide® Kids
4. Slide the stocking over the foot, ensuring that the heel of the stocking sits in the right place
5. Make sure that the light green loop sticks out above the stocking and can be grasped by the child at the top of the stocking
6. Stand at the side or just behind the child and pull together the light green loop to remove the Magnide® Kids from the stocking

Watch our **video instructions** on [uqr.to/MgnKidsEN](http://uqr.to/MgnKidsEN)

#### Preparation to re-use Magnide® Kids:

- I Fix the top of the loop to the top of the product by using the magnet
- II Turn over the product and place it on a flat surface
- III Fold both sides with magnets to the center, so that the magnets make contact. The orange stitching is now visible on the outside
- IV Close the orange flap over the product so that the magnets make contact

#### SIZING

REF	Size	Shoe size EU	Shoe size UK	Shoe size US
E01712	Small	≤ 36	≤ 3,5	≤ 4,5

Watch our **instruction video** here: [uqr.to/MgnKidsEN](http://uqr.to/MgnKidsEN) or scan this QR code:

More information on [www.slide-solutions.com](http://www.slide-solutions.com)

Magnide® Kids from Arion Slide Solutions minimizes friction between compression garment and skin thanks to the product's unique sliding properties provided by the special material it is made of and by its coating. The friction reduction is particularly helpful to slide the garment over the heel area, which is the broadest part of the foot.

#### EASY, SAFE AND SIMPLE SOLUTION

- Reduces physical strain for both compression wearers and caregivers
- May contribute to a longer life for the garment and to maintaining the original compression longer by preventing damage from fingers or nails and avoiding overstretching and distortion
- Promotes independence of compression wearers and supports therapy compliance

#### COMPOSITION

- Product: Polyamide 67%, Polyester 15%, Polypropylene 10%, Magnets 5%, Polyurethane 2%, Silicone 1%
- Package: Polypropylene (PP)

#### CONTRAINDICATIONS

- Do not use the product if you are allergic to one of its component (see composition).
- Open wounds should be covered with an appropriate dressing before using the product.

#### ⚠️ WARNINGS

- **The Magnide® contains several magnets.** Keep this product away from pacemakers and defibrillators. The magnetic field may interfere with the proper operation of this equipment (minimum distance = 30 cm / 11.8").
- **Suffocation hazard:** Keep the product away from babies, children, and pets.
- **Slipping hazard:** Do not step on the product. Highly slippery material.
- **For external use only /** Intended for skin contact. Seek medical attention if irritation develops.

#### IMPORTANT

- Please **read the instructions for use carefully.** If you have any questions, please contact your reseller.
- **Please keep the package and the instructions for use during the time of use of the product.**
- All **serious incidents** that occur when using the product **should be reported** to the manufacturer and the competent supervisory authority.
- When using the product it is important to **use the right product size** for the user. Using an incorrect size will prevent the product from working properly, and it may become damaged or wear out too quickly.
- **Use the product on a single person.** If the product needs to be used on more than one person, disinfect the product before with a 70% ethanol solution.

#### STORAGE

Keep away from direct sunlight or other heat sources.

#### CARE INSTRUCTIONS






- Wash by hand, at max. 30°C/86°F.
- Do not bleach.
- Do not tumble dry. Dry the product only with a towel.
- Do not iron.
- Do not dry clean.
- Disinfection possible with a 70% ethanol solution.

#### GUARANTEE

This product is guaranteed for a period of six months from the date of purchase against defects resulting from faulty material or construction. Within the guarantee period, the product will be exchanged free of charge in the case of manufacturing and/or material defects. Any exchange under the guarantee will only be carried out if it can be demonstrated by means of a purchase receipt that the date on which the complaint was lodged falls within the guarantee period. The guarantee does not apply if the defect is due to improper use of the product or use on several patients. To use this service you need to contact your supplier. You are asked to return the product washed and to follow the care instructions.

#### RECOMMENDATION FOR USE

The manufacturer recommends replacing the product six months after the first use if the product is used regularly. You can write the date of first use on the back of this document.

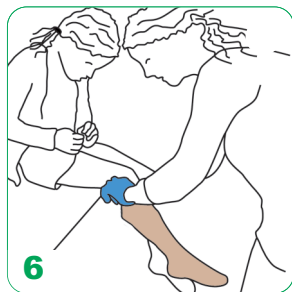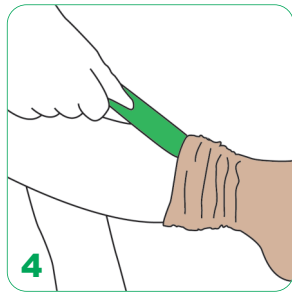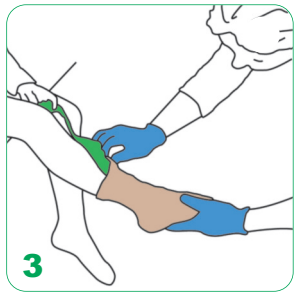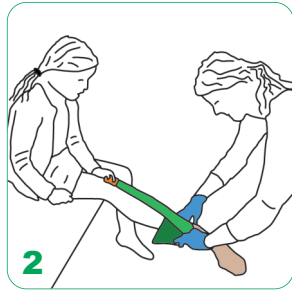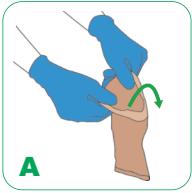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Arion Slide Solutions B.V.