eakin dot

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1. Preparation:

1. **Preparation:** Ensure skin is clean and dry before application (1a).
If the pouch is pre-cut, please proceed to step (2). If the pouch needs to be cut to size, use the enclosed measuring guide cutting guide on the backsheet cut a middle to fit the exact measurements. The measuring guide is available on the inside of the pouch or on the backsheet cut a middle to fit the exact measurements. The measuring guide is available on the inside of the pouch or on the backsheet cut a middle to fit the exact measurements. The measuring guide is available on the inside of the pouch or on the backsheet cut a middle to fit the exact measurements.

2. **Application:** Peel the protective film off the backsheet (2a). Starting at the bottom of the backsheet, press against your skin, smoothing upwards and outwards, to ensure airtight seal. The adhesive on the backsheet will adhere to the skin. Smooth the adhesive on the backsheet to ensure airtight seal. The adhesive on the backsheet will adhere to the skin. Smooth the adhesive on the backsheet to ensure airtight seal.

3. **Emptying disposable pouches:** Fold both legs of the waste trap and inside collar (3a). To make a larger opening for emptying pull the waste trap and inside collar (3b). For emptying pull the waste trap and inside collar (3b). For emptying pull the waste trap and inside collar (3b). For emptying pull the waste trap and inside collar (3b).

4. **Removal of the pouch:** Peel down to remove (4a). Peel down to remove (4a). Peel down to remove (4a). Peel down to remove (4a).

5. **Dispose:** Dispose of waste products in a way that is safe and does not cause any harm to the environment. Dispose of waste products in accordance with local guidelines.

2. Application:

2. **Application:** Peel the protective film off the backsheet (2a). Starting at the bottom of the backsheet, press against your skin, smoothing upwards and outwards, to ensure airtight seal. The adhesive on the backsheet will adhere to the skin. Smooth the adhesive on the backsheet to ensure airtight seal. The adhesive on the backsheet will adhere to the skin. Smooth the adhesive on the backsheet to ensure airtight seal.

3. **Emptying disposable pouches:** Fold both legs of the waste trap and inside collar (3a). To make a larger opening for emptying pull the waste trap and inside collar (3b). For emptying pull the waste trap and inside collar (3b). For emptying pull the waste trap and inside collar (3b).

4. **Removal of the pouch:** Peel down to remove (4a). Peel down to remove (4a). Peel down to remove (4a). Peel down to remove (4a).

5. **Dispose:** Dispose of waste products in a way that is safe and does not cause any harm to the environment. Dispose of waste products in accordance with local guidelines.

3. Application:

3. **Application:** Peel the protective film off the backsheet (3a). Starting at the bottom of the backsheet, press against your skin, smoothing upwards and outwards, to ensure airtight seal. The adhesive on the backsheet will adhere to the skin. Smooth the adhesive on the backsheet to ensure airtight seal. The adhesive on the backsheet will adhere to the skin. Smooth the adhesive on the backsheet to ensure airtight seal.

4. **Removal of the pouch:** Peel down to remove (4a). Peel down to remove (4a). Peel down to remove (4a). Peel down to remove (4a).

5. **Dispose:** Dispose of waste products in a way that is safe and does not cause any harm to the environment. Dispose of waste products in accordance with local guidelines.

4. Application:

4. **Application:** Peel the protective film off the backsheet (4a). Starting at the bottom of the backsheet, press against your skin, smoothing upwards and outwards, to ensure airtight seal. The adhesive on the backsheet will adhere to the skin. Smooth the adhesive on the backsheet to ensure airtight seal. The adhesive on the backsheet will adhere to the skin. Smooth the adhesive on the backsheet to ensure airtight seal.

5. **Dispose:** Dispose of waste products in a way that is safe and does not cause any harm to the environment. Dispose of waste products in accordance with local guidelines.

5. Application:

5. **Application:** Peel the protective film off the backsheet (5a). Starting at the bottom of the backsheet, press against your skin, smoothing upwards and outwards, to ensure airtight seal. The adhesive on the backsheet will adhere to the skin. Smooth the adhesive on the backsheet to ensure airtight seal. The adhesive on the backsheet will adhere to the skin. Smooth the adhesive on the backsheet to ensure airtight seal.

6. **Dispose:** Dispose of waste products in a way that is safe and does not cause any harm to the environment. Dispose of waste products in accordance with local guidelines.

6. Application:

6. **Application:** Peel the protective film off the backsheet (6a). Starting at the bottom of the backsheet, press against your skin, smoothing upwards and outwards, to ensure airtight seal. The adhesive on the backsheet will adhere to the skin. Smooth the adhesive on the backsheet to ensure airtight seal. The adhesive on the backsheet will adhere to the skin. Smooth the adhesive on the backsheet to ensure airtight seal.

7. **Dispose:** Dispose of waste products in a way that is safe and does not cause any harm to the environment. Dispose of waste products in accordance with local guidelines.

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