Natura™TM

INSTRUCTIONS FOR USE • INSTRUCCIONES DE USO • MODE D'EMPLOI • INSTRUÇÕES DE USO •
使用説明 • 使用説明書 • GEBRAUCHSANWEISUNG • INSTRUZIONI D'USO •
INSTRUCTIONS D'UTILISATION • GEBRUIKSINSTRUCTIE • BRUKSANVISNING • KÄYTTÖOHJEET
• BRUGSANVISNING • BRUKSANVISNING • ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ • INSTRUKCJA STOSOWANIA •
HASZNÁLATI UTASÍTÁS • NÁVOD K POUŽITÍ • NÁVOD NA POUŽITIE • UPUTE ZA UPORABU •
NAVODILA ZA UPORABO • ИНСТРУКЦИЯ ПО ПРИМЕНЕНИЮ • INSTRUCIUNI DE UTILIZARE •
KULLANIM TALİMATLARI • KASUTUSJUHEND • UPUTSTVO ZA UPOTREBU •
LIETOSĀNAS PAMĀCĪBA • NAUDOJIMO INSTRUKCIJOS

ENGLISH

Natura™  **Two-Piece Urostomy Pouch**

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION
Natura™  Urostomy Pouch is part of a two-piece ostomy system. This pouch must be used with the same size Natura™, Natura™  , Sur-Fit Natura™, and Combihesive Natura™ skin barriers.

PRECAUTIONS AND OBSERVATIONS

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

INDICATION

For the management of stomal output from a urostomy.



DIRECTIONS FOR USE

Note: Refer to the skin barrier instructions for use for applying a pouch to the skin barrier, removing the pouch from the skin barrier and disposing of the pouch.

1. Drainage of the Pouch

2. Connection to Night Drainage System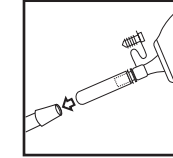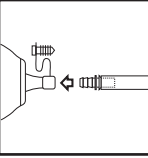

Note: Use only the connector provided.

Made in Dominican Republic

© 2021 ConvaTec Inc.™ indicates a trademark of ConvaTec Inc. NATURA, Sur-Fit, and Sur-Fit Natura are registered trademarks in the United States.

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© 2021 ConvaTec Inc.™ indikerar trademarks tillhörande ConvaTec Inc. NATURA är ett registrerat varumärke tillhörande ConvaTec Inc. i United States.