**Silikonpflaster**

**Silicone Plasters**

**Siliconenpleisters**

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### **EN: Instructions for using Silicone Plasters**

Introduction: Zensetiv silicone plasters are self adhesive dressings to protect a stoma after irrigation or a continent stoma. The central island of viscose material will absorb and retain stoma secretion. The silicone coated polyurethane foam will adhere the plaster to the skin, is non irritant and easy to remove, while preventing leakage.

- Clean the skin around the stoma with water if necessary
- Avoid the use of alcohols, sprays or creams etc.
- Remove the release film that covers the adhesive (picture 2)
- Take the silicone plaster on the side with the blue arrows on it (picture 3)
- Apply the silicone plaster from bottom to top (picture 4)
- Remove the strip with the blue arrows, at the top (picture 5)
- Secure the plaster by rubbing it gently

**Caution:** the silicone plasters are for single use only. Re-use can lead to cross-contamination. Zensetiv silicone plasters should not be used by anyone with a known sensitivity to either silicone, polyurethane or viscose. The silicone plaster should only be used after irrigation of the stoma or when the stoma is known to be continent.

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