Clickfine®
AutoProtect™

swiss made +

AUTO PROTECT

Brugsanvisning

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Instructions for use

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Käyttöohje

Mode d'emploi

Istruzioni d'uso

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Bruksanvisning

Bruksanvisning

da

de

en

es

fi

fr

it

nl

no

sv

0.25 mm × 5 mm – 31G
0.33 mm × 8 mm – 29G

YPSOMED
SELF CARE SOLUTIONS

2010/32/EU §4, Art. 7 ✓

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Warnings:

⚠ Follow the **instructions for use of your injection pen** and use the mylife™ Clickfine® AutoProtect™ pen needles (hereinafter "mylife™ Clickfine® AutoProtect™ pen needle") for your injection pen only after consultation with your physician and/or healthcare professional.

⚠ Avoid **infections**. Do not use the mylife™ Clickfine® AutoProtect™ pen needle if the individual packaging **[A, B, E]** is damaged, contaminated or has been opened by someone other than you. Do not use the mylife™ Clickfine® AutoProtect™ pen needle after the expiration date **[F]** on the packaging. Before each injection, clean the injection site thoroughly according to your physician's and/or healthcare professional's instruction. If these instructions are not followed, there is a risk of infection. Seek immediate medical attention if you have any health concern.

⚠ The mylife™ Clickfine® AutoProtect™ pen needle cannot be used if the red locking indicator **[H]** is visible. Use a new mylife™ Clickfine® AutoProtect™ pen needle.

⚠ Ensure **correct delivery of medication**. Before injection, check that the mylife™ Clickfine® AutoProtect™ pen needle allows free flow of medication, following the instructions for use for your injection pen (priming test). This will prevent incorrect dosage caused by air in the cartridge or a clogged needle. Use a new

mylife™ Clickfine® AutoProtect™ pen needle if priming is not successful. Inject slowly according to the instructions of your physician and/or healthcare professional and of the injection pen manufacturer, and wait for a few seconds after the medication has been completely discharged before withdrawing the injection pen from the injection site. Check after each injection according to your physician's and/or healthcare professional's instructions whether medication drips from the mylife™ Clickfine® AutoProtect™ pen needle or injection pen, or exudes from the injection site. If the medication drips from the mylife™ Clickfine® AutoProtect™ pen needle or injection pen, or exudes from the injection site, follow your physician's and/or healthcare professional's instructions regarding the handling of potential under-dosage. Seek medical attention if you have any health concern.

⚠ Avoid **breaking the mylife™ Clickfine® AutoProtect™ pen needle**. Do not change the angle of the injection pen after inserting the pen needle **[D]** into the skin. Changing the angle can result in bending or breaking of the mylife™ Clickfine® AutoProtect™ pen needle. A broken pen needle can remain stuck in the body or remain completely under the skin. Seek immediate medical attention if a broken mylife™ Clickfine® AutoProtect™ pen needle becomes stuck in the body or remains under the skin.

Instructions for the pictures (see page 4 & 5):

- 1 Remove the protective foil.
- 2 Do not hold at an angle when clicking into place.
- 3 Click (or screw) the mylife™ Clickfine® AutoProtect™ onto the injection pen, **keeping it straight**.
- 4 Screw to secure pen needle. Firm seating is ensured when screwed on, even without a perceptible limit stop.
- 5 Remove the protective cap. The red locking indicator should not be visible.
- 6 Before injection perform a priming test, following the instructions for use for your injection pen. Set the dose in accordance with the instructions of your physician and/or healthcare professional.
- 7 Do not put any pressure on the dose button during needle insertion.
- 8 Insert the injection pen at a **90° angle** to the desired injection site, keeping it vertical. In order to avoid injecting into the muscle, it may be necessary to use a lifted skin fold.
- 9 Exert a gentle pressure on the injection pen until the blue protective sleeve [C] is no longer visible.
- 10 Slowly inject the insulin according to the instructions of your healthcare professional: to do this, press dosage button. Allow the pen needle to remain in the fatty tissue (slowly count to 10).
- 11 Remove the injection pen from the injection site, keeping it straight. The blue protective sleeve locks automatically and the red locking indicator becomes visible.
- 12 Unscrew the mylife™ Clickfine® AutoProtect™ pen needle from the injection pen.
- 13 Dispose of the needle safely, in accordance with local regulations.

Caution:

- Change the injection site regularly in accordance with your physician's and/or healthcare professional's advice. If this is not done, tissue hardening may result.
- The mylife™ Clickfine® AutoProtect™ pen needle is compatible with all the injection pens listed on the pen needle box. Please contact us if you have any questions about using mylife™ Clickfine® AutoProtect™ pen needles with other injection pens.
- The Novo Nordisk Function Check cannot be performed with the mylife™ Clickfine® AutoProtect™ pen needle.
- Professional healthcare personnel should follow the specific directions for healthcare personnel. To obtain the specific directions, please consult a physician.

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- FI** Ypsomed Oy // Eteläinen Salmitie 1 // 02430 Masala/
- FR** Ypsomed S.A.S // 44 rue Lafayette // 75009 Paris/
- IT** Ypsomed Italia S.r.l. // Via Santa Croce, 7 //
- SE** Ypsomed AB // Adolfsbergsvägen 31 // 168 66 Bromma/
- UK** Ypsomed Ltd. // 1 Park Court // Riccall Road // Escrick //
Ypsomed AG