
Absorbing Sachets

REF A2020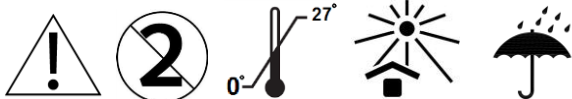

Keep out of reach of children.
 Außer Reichweite von Kindern aufbewahren.
 Buiten het bereik van kinderen houden.
 Opbevares utilgængeligt for børn.
 Mantener fuera del alcance de los niños.
 Tenir hors de la portée des enfants.
 Glabājiet bērniem nepieejamā vietā.
 A nu se lăsa la îndemâna copiilor.
 Förvara utom räckhåll för barn.
 Να φυλάσσεται μακριά από παιδιά.
 Mantenha fora do alcance de crianças.
 Uchovávejte mimo dosah dětí.
 Uchovávejte mimo dosahu detí.
 Trzymać poza zasięgiem dzieci.
 يحفظ بعيدا عن تناول الأطفال.
 Čuvajte van dohvata djece.
 Čuvati dalje od djece.
 Laikyti vaikams nepasiekiamoje vietoje.
 Oppbevares utilgjengelig for barn.
 Hranite zunaj dosega otrok.

Do not swallow.
 Nicht verschlucken.
 Niet inslikken.
 Må ikke indtages.
 No lo tragues.
 Ne pas avaler.
 Nenorīt.
 Nu inghiti.
 Får ej sväljas.
 Μην καταπίνετε.
 Não engula.
 Nepolykejte.
 Neprehltajte.
 Nie połykać.
 لا ابتلاع.
 Nemojte gutati.
 Ne progutati.
 Nevalgyti.
 Må ikke svelges.
 Izdelka ne smete zaužiti.

GB: Instructions for Use

Features and Intended Use

Highly absorbent granules to thicken the output collected in stoma pouches and to help neutralise odour.

The “film layer” surrounding the granules dissolves as soon as it comes into contact with a stool. The highly absorbent granules then release to thicken the stool. The granules will neutralise odours.

Materials/Ingredients

Super absorbent polymers.

Application

- If you use a colostomy pouch (closed), insert one sachet containing absorbent granules, into a new stoma pouch before application.
- If you use an ileostomy pouch (open), empty the pouch and clean the closure, then insert a sachet through the closure of the pouch.

Disposal of the device after use

This product can be used in colostomy pouches and may be disposed of in the same way as you usually dispose of your pouches. When using an ileostomy pouch the contents including the granules may be emptied from the pouch and flushed away in the toilet.

Warnings and Precautions

- Each product is for single use only. Do not re-use. Re-use can lead to cross-contamination.
- Use only as directed by your healthcare professional.
- Read the instructions prior to use.
- Discontinue use and contact your healthcare professional if you experience any allergic reactions or problems when using the product.
- For external use only.
- Keep out of reach of children.
- If the contents of the sachet comes into contact with the eyes, rinse thoroughly with clean water and consult a doctor if irritation occurs.
- Do not swallow. If product is swallowed there may be a risk of suffocation. Do not induce vomiting. Rinse mouth with water and seek medical assistance immediately. Monitor until all granules are cleared from mouth and throat.
- Non-toxic.

Contraindications

These devices should not be used by anyone with a known sensitivity to the materials or ingredients listed in the section 'Materials/Ingredients'.

Storage conditions

Store in a cool dark dry place away from moisture. Best stored with a guidance of temperature from 0°C to 27°C as is noted on the product label.

Serious Incidents

Should a serious incident arise as a result of using the product please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

Shelf life

Use by expiry date printed on the label.

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