3. Applying the Skin Barrier (Figure 2)
3. Remove the release liner from the skin barrier.
- 3.2 Peel the opening of the skin barrier over the stoma.
- 3.3 Apply the skin barrier against the skin around the stoma for 30 seconds.
- 3.4 When the remaining adhesive on the fabric dries, press it against the stoma.
- 3.5 Smooth in place and remove wrinkles.
- 3.6 Wear as long as comfortable and secure. Wear time may vary.

4. Applying the Pouch (Figure 3)
- 4.1 Hold the Accordion Flange™ with three fingers of both hands.
- 4.2 Peel the Accordion Flange™ and the pouch flange (Figure 4)
- 4.3 Starting at the bottom, apply gentle pressure around the circumference of the flange until they snap in place and feel secure.
- 4.5 Try to open the pouch to ensure a secure connection (Figure 5)

5. Closing the Pouch (Figure 4)
- 5.1 Drainable Pouches with InvisiClose™ Tall Closure (Figure 6) Fold the call closure beginning with the tail by twisting on both ends of the outlet end-strips with fingers; drain pouch.
- 5.2 Drainable Pouches with Tall Call (Figure 12)
- 5.3 Urinary Pouches with AccuSeal™ Tap (Figure 13)
- 5.4 Urinary Pouches with Fold over Tap (Figure 14)
- 5.5 Urinary Pouches with Soft Tap (Figure 10)

6. Emptying the Pouch (Figure 1)
- 6.1 Drainable Pouches with InvisiClose™ Tall Closure (Figure 11) Hold the pouch closing and open the tail by twisting on both ends of the outlet end-strips with fingers; drain pouch.
- 6.2 Drainable Pouches with Tall Call (Figure 12)
- 6.3 Urinary Pouches AccuSeal™ Tap (Figure 13)
- 6.4 Urinary Pouches with Fold over Tap (Figure 14)
- 6.5 Urinary Pouches with Soft Tap (Figure 10)

7. Connection to Night Drainage System (Figure 1)
- 7.1 Connect to Tap (Figure 15)
- 7.2 Connect to Night Drainage System (Figure 16)
- 7.3 Seal Tap (Figure 17)

8. Information about Pouches with Filters (Figure 1)
- 8.1 Cover the filter with the sticker provided when bathing, swimming, etc.
- 8.2 Remove the filter when water activity.

9. Removing the Pouch (Figure 18)
- 9.0. Removing the Skin Barrier (Figure 19)

10. Removing the Skin Barrier (Figure 19)
- 10.1. Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

11. Disposal If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.
- 11.1 After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

12. Removal of the Skin Barrier (Figure 19)
- 12.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

13. Removal of the Skin Barrier (Figure 19)
- 13.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

14. Removal of the Skin Barrier (Figure 19)
- 14.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

15. Removal of the Skin Barrier (Figure 19)
- 15.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

16. Removal of the Skin Barrier (Figure 19)
- 16.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

17. Removal of the Skin Barrier (Figure 19)
- 17.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

18. Removal of the Skin Barrier (Figure 19)
- 18.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

19. Removal of the Skin Barrier (Figure 19)
- 19.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

20. Removal of the Skin Barrier (Figure 19)
- 20.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

21. Removal of the Skin Barrier (Figure 19)
- 21.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

22. Removal of the Skin Barrier (Figure 19)
- 22.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

23. Removal of the Skin Barrier (Figure 19)
- 23.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

24. Removal of the Skin Barrier (Figure 19)
- 24.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

25. Removal of the Skin Barrier (Figure 19)
- 25.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

26. Removal of the Skin Barrier (Figure 19)
- 26.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

27. Removal of the Skin Barrier (Figure 19)
- 27.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

28. Removal of the Skin Barrier (Figure 19)
- 28.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

29. Removal of the Skin Barrier (Figure 19)
- 29.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

30. Removal of the Skin Barrier (Figure 19)
- 30.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

31. Removal of the Skin Barrier (Figure 19)
- 31.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

32. Removal of the Skin Barrier (Figure 19)
- 32.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

33. Removal of the Skin Barrier (Figure 19)
- 33.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

34. Removal of the Skin Barrier (Figure 19)
- 34.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

35. Removal of the Skin Barrier (Figure 19)
- 35.1 Disposal This device is for single-use only and should not be re-used. You may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

©2021 ConvaTec Inc.
 1 Your hands and the Accordion Flange™ system allow the flange to be placed with the same size Natura™, Natura™-Sur-Fit™ Natura™ or Combishave™ Natura™ pouches.

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