■ Black

148 mm

**carefix**<sup>®</sup>  
Creating functional comfort

# CarePocket<sup>®</sup> Premium

1

2

3

**EN**

## Leg Bag Holder

### Indication:

Medical device for retention of urine leg bag.

### Features:

CarePocket<sup>®</sup> Premium provides a pocket for retention of a urine leg bag. It has a slit at the bottom for the valve of urine bags with bottom tap opening. The sleeve can contain urine leg bags of volumes up to 750 ml/24 oz. Suitable for use on both thigh and calf. With outer seam.

### Precautions:

It is recommended to empty the urine bag when two-thirds full. If you have any questions regarding use and/or correct fitting, or if any unexpected discomfort or skin irritation occurs, please consult your healthcare professional.

### Instruction for use:

1. Pull on the leg bag holder. Ensure that the two coloured lines are at the top.
2. Pull down the outer layer of the sleeve and place the leg bag in the pocket. Pull the tap through the hole at the bottom of the sleeve. Pull up the outer layer of the sleeve.
3. The CarePocket<sup>®</sup> Premium is in place.

Art. No. 5080

60°C/140°F

### Tytex Customer Service:

E-mail: [customerservice@tytex.com](mailto:customerservice@tytex.com)  
International: +45 9660 4200; +49 2262 762914  
Americas: +1-855-569-6131

Tytex A/S has been certified according to ISO 9001, ISO 14001 and ISO 13485  
Made in Slovakia by: TYTEX A/S · Industrivej 21 · DK-7430 Ikast · [www.tytex.com](http://www.tytex.com)

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