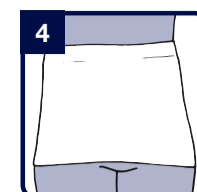
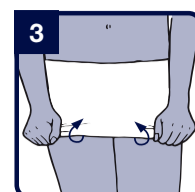
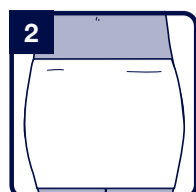
**PLEASE KEEP THIS INSTRUCTION FOR FURTHER REFERENCE**

**EN:**

**Indication:**

Corsinel ostomy and hernia support tube is specially designed for ostomy, parastomal hernia, umbilical hernia, epigastric hernia and abdominal scar hernia. The tube helps fixate the ostomy pouch, provides discretion and a smooth appearance. Furthermore, the tube may reduce unpleasant smells and noises from a stoma. The tube is also developed to provide abdominal support of the above hernia types and may reduce the risk of hernia development. The mobility from the tube offers relief, alleviation, assists support and counteracts sense of gravity, thereby providing pain relief. The Corsinel tube is available in three heights: 15, 20 and 26 cm. For recommendation on height and use, please contact your doctor or nurse.

**Features:**

The Corsinel tube supports a hernia. The silicone anti-roll pattern on the elastic bands prevents the tube from rolling. The 20 and 26 cm models have a lower back, which improves the anatomic fit. The tube is made of air- and moisture permeable material. There is no difference between up and down.

**How to measure yourself:**

**With a hernia:**

Measure yourself when lying down and when the hernia has retracted if possible. Measure yourself at the widest abdominal circumference covered by the tube.

**Without a hernia:**

Measure yourself in standing position and at the widest abdominal circumference covered by the tube.

**NOTE:**

Sizes overlap; if abdominal measurement is between two sizes, choose the smaller size if you prefer an extra tight-fitting tube, and the larger size if you prefer the tube less tight. Ensure that the tube feels comfortable to wear, when sitting and standing.

**Instructions for use:**

1. Turn the tube inside out and apply it down or underwear. If possible, apply the tube when lying in knee sitting. If you have a 20 or 26 cm model, make sure that the low-cut part is located at the back.
2. Pull the tube up so that the upper, elastic waistband is positioned slightly above the hips.
3. The tube is then easily applied around the abdomen by pulling the lower elastic waistband up, so the tube rolls into the correct position.
4. Make sure that the tube sits smoothly and has no folds.

**Precautions:**

Make sure the tube is not compressed with the tube/during lung functions. We advise against using the tube during pregnancy, by acute abdomen, abdominal compartment syndrome, presence of ascites fluid or general discomfort/pain in the abdominal region. Please consult your doctor if you suffer from chronic obstructive pulmonary disease, if you experience unexpected discomfort, stoma or skin irritation or if you have any questions regarding use. If urgent need for removal of the product, open with scissors.

**Care handling:**

Wash with similar colours in a washing bag. Use of chlorine-based bleach or chlorine-based disinfectant will decrease durability and function of the product.

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