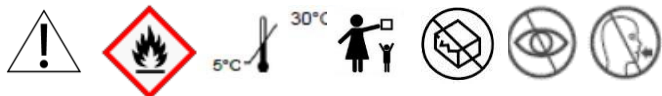
**ZenSetiv Adhesive Remover****REF ZSA2011****EN: PRODUCT DESCRIPTION**

ZenSetiv Adhesive Remover is a specially formulated solution that ensures atraumatic removal of adhesive devices.

Indications:

For the safe and pain free removal of all medical adhesive devices and skin residues including:

- Adhesive tapes
- Wound Dressings
- Stoma Flange
- other Adhesive Products

Adhesive remover is particularly suited for removing difficult residues which build up on the skin over time.

Contraindications:

- ZenSetiv Adhesive Remover should not be used on skin where infection is present.
- Do not use this product if there is a known allergic sensitivity to any of the ingredients.

**Warnings:**

ZenSetiv Adhesive remover in liquid form is flammable; use in a well ventilated area. Do not use around flames and sources of ignition. Keep out of reach of children. Do not use if pack is damaged or open. Avoid contact with eyes. For external use only. Do not swallow.

Directions for Use:**When removing adhesive dressings:**

- Lift up edge of plaster or hydrocolloid and apply ZenSetiv Adhesive Remover under the product while lifting off.
- Peel away the product slowly while continuing to apply more adhesive remover as the product begins to lift.
- After wiping clean the surrounding skin area, a new product can be applied if required.

For removing adhesive residue on the skin:

- Apply adhesive remover to the affected area.
- Allow the solution to rest on the adhesive for 1-2 minutes.
- Gently wipe clean the skin area.
- Re-apply for extremely difficult residues.

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