Esseeem™ Flex Convex

ná sára som možlíg. Klipp upp stómhléttí háudskyddsplattan til stómias dómara eller nágot stómí (figur 1).

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1. **PREPARATION**
Clean the skin around your stoma with water and a soft cloth. Pat the area dry. Do not use oil-based products as they may interfere with the adhesion of the skin barrier.

Please read these instructions for use carefully, as they contain important information about the use of our Esteeem™ + Flex Convex products.

PRODUCT DESCRIPTION
Esteeem™ + Flex Convex pouches are one-piece closed, drainable or urostomy pouches with a pre-cut or cut-to-fit skin barrier.

INTENDED USE: Management of stoma output.

INDICATIONS FOR USE: For use with a colostomy, ileostomy or urostomy.

PRECAUTIONS AND OBSERVATIONS
This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be optimal for intended use.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

For cut-to-fit pouches, make sure the opening you cut is not too large. An ill-fitting pouch can cause skin irritation and inflammation. Be very careful not to increase the opening size past the outer ring marked on the water or to the edge of or exposing the inner plastic convex disc.

Please note that different pouch types are appropriate for different stoma types. Please consult your healthcare professional on the most appropriate pouch type for your stoma prior to use.

Important: A healthcare professional should assess the need for skin barriers with convexity prior to use.

DIRECTIONS FOR USE

1. **Preparation**
Clean the skin around your stoma with water and a soft cloth. Pat the area dry. Do not use oil-based products as they may interfere with the adhesion of the skin barrier.

Please make sure you know the diameter of your stoma. Esteeem™ + Flex Convex is available in pre-cut and cut-to-fit options.

For cut-to-fit pouches you can use the cutting guide on the water here provided. The hole in the adhesive water should fit the stoma as closely as possible. Cut the stoma hole in the water to the diameter of your stoma or a little bigger (figure 1)

2. **Use of the Esteeem™ + Flex Convex system**
Partially inflate the pouch by blowing to separate the inner layers of the pouch. Warm the skin barrier with the palm of your hand for a few seconds to dry adhesive. Remove the plastic covering from the skin barrier (figur 2). Try to avoid touching the adhesive area too much with your hands.

Hold the skin barrier below your stoma. Beginning from the bottom carefully adhere the barrier to the skin around your stoma (figure 3).

Smooth over the back of the adhesive area with your fingers and press carefully but firmly around your stoma (figure 4).

3. **Advice for use of a drainable pouch with Safe Seal closure**
For those with an ileostomy or colostomy there is a type of pouch that can be drained. The tail closure cleaned and marked as follows.

Please note the following advice for using a drainable pouch with Safe Seal closure:

Make sure the bottom of the pouch is securely fastened. Fold up the neck of the pouch and continue for a total of 5 folds until the two parts of the fastener are in contact with each other. Press firmly together to gain a secure seal (figure 5).

Apply the pouch as instructed. Pull the tail under the closure and empty the pouch.

4. **Advice for using a urostomy pouch**
Make sure the bottom of the pouch is securely fastened. Apply the pouch as instructed. Use the supplied adaptor to connect the urostomy pouch to a night drainage pouch or leg bag if necessary (figure 6).

5. **Information about pouches with filters**
There is a built in filter in closed and drainable pouches through which gases are vented and neutralized. The filter sticker provided serves as an effective protection against moisture (e.g. when bathing, showering) and ensures thereby the functional performance of the filter.

Cover the filter with the sticker when bathing, swimming, etc (figure 7). Remove after water activity.

6. **Pouch removal and disposal**
To remove an old pouch for disposal, gently peel downwards while holding the skin above the water with your free hand (figure 8). This helps avoid skin irritation and makes the removal easier.

Empty the contents of the pouch into the toilet.

Never flush the pouch down the toilet.
After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

Made in Germany

© 2021 ConvaTec Inc.

™ All trademarks are the property of their respective owners. ESTEEM is a registered trademark of ConvaTec Inc. in the United States.

1. **PREPARATION**
Clean the skin around your stoma with water and an mjuk servett. Klappa området torrt. Anvánd ítt óþessara dómara produktora eftir með sin þærri háttir húddskyddsplattan með konvexttí filter og andentligt inntúg.

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Esteem™ est une marque déposée de ConvaTec Inc. enregistreé aux États-Unis.