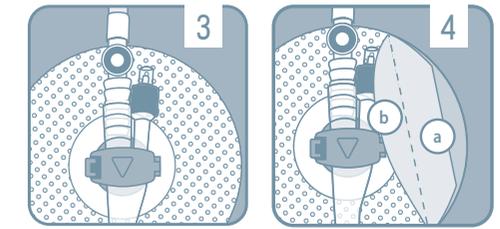
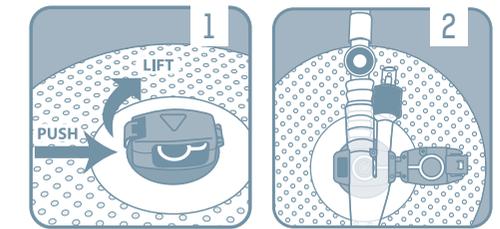
Pantone 5425

ugofix™ gentle

3004-IFU-Oct18-V2

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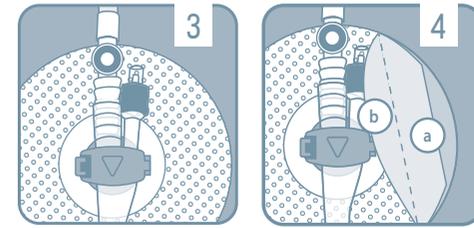
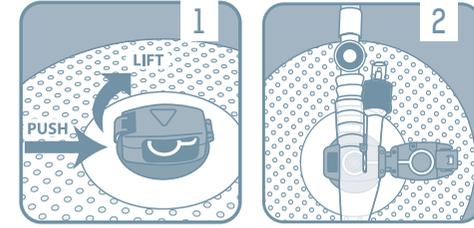
ugofix™ gentle

3004-IFU-Oct18-V2

EC REP MT Promed Consulting GmbH
Altenhofstrasse 80
66386 St. Ingbert Germany

REF **3004**     CE

GBR Ugo Fix Gentle Catheter Clip

At **Optimum Medical** we understand the need for secure and reliable catheter fixation. We have developed the **Ugo Fix Gentle** to offer comfortable fixation, giving the patient the confidence to move around freely. The **Ugo Fix Gentle** provides fixation of 2 way indwelling urinary catheters at the bifurcation of the catheter. It enables fixation of urethral or suprapubic catheters which are attached to catheter valves or drainage bags. The **Ugo Fix Gentle** is one size to fit all 2 way indwelling catheter sizes and materials.

Benefits of using the Ugo Fix Gentle

The **Ugo Fix Gentle** reduces the risk of trauma and pain caused by the catheter pulling or repeatedly rubbing on the bladder neck and urethra. Each **Ugo Fix Gentle** can be used for up to 7 days. Only change it sooner if it is damaged or soiled. You can shower whilst wearing the **Ugo Fix Gentle**.

Revolving clip

- Revolving clip motion allows the catheter to move in natural unison with the body
- Raised arrow on top of clip points towards the bladder
- Internal silicone bonding provides grip to catheter surface
- Holds catheter away from skin to prevent skin marking

Soft silicone re-adhesive gel pad

- Atraumatic removal without use of solutions to dissolve adhesive
- No sticky residue
- Conforms to all skin types through multiple points of contact
- Accun résidu collant
- S'adapte efficacement à tous types de peau sur des points de contact multiples
- Can be repeatedly applied to same skin area
- Re-adhesive tack enables easy repositioning
- No complex skin preparation required

Instructions for use

It is important that the catheter is fixed into the **Ugo Fix Gentle** prior to applying to the skin.

Before applying to the skin

Please note: the **Ugo Fix Gentle** is designed to fix and secure the catheter. If the catheter is attached to a urine drainage bag, it is advisable to use additional support for the weight of the urine bag.

- | | | |
|--|--|--|
| <p>1. Open clip</p> <ul style="list-style-type: none"> • Push and lift the catch to open the Ugo Fix Gentle lid • You will sense a click | <p>2. Insert catheter</p> <ul style="list-style-type: none"> • Place the bifurcation part of the catheter into the Ugo Fix Gentle (it can only fit one way) | <p>3. Close lid</p> <ul style="list-style-type: none"> • Once in place, close the Ugo Fix Gentle lid by gently pressing it down • You will sense a click |
|--|--|--|

Application to skin

4. Now the catheter is fixed into the clip, the **Ugo Fix Gentle** can be applied to the skin. Ensure the skin is clean and dry before application.
- Remove backing (A) and apply to skin in desired position
 - Remove backing (B) and smooth in place
 - Finally ensure the catheter is not pulling when stood up and you are able to move around comfortably with it in place

WARNING: Keep out of reach of children!

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