

**EN Instructions for use**

The Accu-Chek FastClix finger pricker is a device for simple, virtually pain-free collection of capillary blood from the fingertips. The finger pricker has 11 different penetration depths so that you can adjust the penetration depth to the texture of your skin. With the release button, you can prime the finger pricker and trigger a lancet in a single step. When a lancet is triggered, it penetrates the skin. You insert a lancet drum with 6 lancets into the finger pricker. You can obtain blood using a new, sterile lancet 6 times before the lancet drum needs changing.

These instructions for use feature the following 3 symbols:

- This symbol indicates a possible risk of injury or of damage to your health.
- This symbol draws attention to actions that could result in damage to the finger pricker.
- This symbol draws your attention to important information.

**The Accu-Chek FastClix finger pricker is intended for patient self-monitoring by an individual person only.** It must not be used to collect blood from more than one person as it does not incorporate any features to guard against cross-infection. There is a risk of infections being transmitted if the finger pricker is used by other people, even by family members, or if healthcare professionals use this finger pricker to obtain blood from different people. This finger pricker is therefore not suitable for professional use in healthcare facilities and institutions.

- Keep the finger pricker with all its components out of reach of children under 3 years. There is a risk of suffocation if small parts (e.g. caps or similar objects) are swallowed.

Only insert Accu-Chek FastClix lancet drums into the Accu-Chek FastClix finger pricker. Other lancets or lancet drums may impair the operation of the finger pricker or damage it permanently.

**Preparation**

- Wash your hands with warm water and soap before you obtain blood (see 1).
- Dry your hands thoroughly. This reduces contamination of the puncture site and stimulates blood flow.

**Inserting the lancet drum**

- You have to insert a lancet drum to be able to obtain blood with the finger pricker.
- Remove the cap from the finger pricker (see 2).

- Insert a new lancet drum – white end first – into the finger pricker (see 2).

The lancet drum is properly inserted when it is completely inside the finger pricker, and only the projecting edge protrudes from the finger pricker at the front (see 3).

You must not insert the lancet drum into the finger pricker and simultaneously press the release button or hold the finger pricker with the release button resting on a surface such as a table top. This could release a lancet and inadvertently cause injury.

- Place the cap back on again. The cut-out section at the edge of the cap must fit around the correspondingly shaped curvature with the penetration depth indicator (silver square) (see 4).

The cap is on correctly when it is pushed back as far as it will go, and you hear and feel it click into place.

The cap can only be correctly placed up to the stop position when the lancet drum is completely inside the finger pricker. Do not use any force to place the cap on as this might damage the cap and the finger pricker. Correct the position of the lancet drum instead.

Each lancet drum contains 6 lancets. The first lancet is now ready to be used. The window showing the number of available lancets shows 6; there are now 6 lancets available (see 5).

Do not remove the lancet drum before all 6 lancets have been used. The lancet drum cannot be re-inserted once it has been removed from the finger pricker.

**Setting the penetration depth**

You can set 11 penetration depths (0.5 to 5.5). The silver square (penetration depth indicator) on the finger pricker indicates the current penetration depth setting. The larger the number, the greater the penetration depth.

If you have no experience with this finger pricker, we recommend setting a low penetration depth, such as penetration depth 2.

- Rotate the cap until the desired penetration depth lines up with the indicator (see 6).

**Obtaining a blood drop**

In general, blood can be obtained from any finger. Certain fingers can be unsuitable if, for example, a skin or fingernail infection is present.

We recommend obtaining capillary blood from the sides of the fingertips as these areas are the least sensitive to pain.

Always use a new lancet when you obtain blood. In doing this, you reduce the risk of infection, and obtaining blood remains virtually pain free.

Only use the finger pricker when the cap is attached. Without the cap, the lancet penetrates too deeply and the puncture can become painful.

In individual cases, the lancet may protrude from the cap after obtaining blood. There is a risk of injury or infection if you or another person touches the protruding lancet. If the lancet protrudes from the cap, carefully set the finger pricker aside. Keep the finger pricker out of reach of other people and contact the Accu-Chek Customer Careline.

- Press the finger pricker firmly against the selected puncture site (see 7).
- Press the release button all the way down (see 8).

The lancet is released and it penetrates the skin. Massage the finger in the direction of the fingertip to encourage a blood drop to form.

The amount of blood that emerges depends on the penetration depth and the pressure used to hold the finger pricker against the skin.

If not enough blood emerges, apply more pressure on the finger pricker when you next obtain blood. If that is not sufficient, increase the penetration depth progressively in addition.

If too much blood emerges, decrease the penetration depth.

Test your blood glucose immediately after you obtained blood, according to the instructions for use for your blood glucose meter.

We recommend protecting the puncture site from contamination afterwards.

**Loading a new lancet**

If you have already used the finger pricker to obtain blood, you will need a new lancet the next time you obtain blood.

- Push the lever at the side of the finger pricker as far as it will go in direction A and then back in direction B (see 9).

The next lancet is now ready to be used.

The window next to the lever shows the new number of available lancets, which has decreased by one, for example, 5 instead of 6.

- Proceed as in the previous step each time you need to change the lancet.

Each time a new lancet is loaded, the number of available lancets shown in the window decreases by one, for example, from 4 to 3. When you have loaded the sixth and last lancet, the window shows 1.

**Replacing a used lancet drum**

When you have used the sixth and last lancet, you need to replace the used lancet drum with a new one.

- Remove the cap from the finger pricker (see 2).
- Remove the lancet drum from the finger pricker (see 10).

When a lancet drum has been used, a red stripe is visible on the white cylinder.

A used lancet drum has a built-in lock that stops you from using it again.

Used lancet drums can be disposed of in household waste if no other regulations apply in your country.

**Storing the finger pricker**

Do not store the finger pricker at extreme temperatures, for example, in a hot car. This can impair the operation of the finger pricker.

**Cleaning and disinfecting the finger pricker and cap**

To prevent infections from being transmitted, you must clean and disinfect the finger pricker and cap regularly:

- At least once a week
- When there is blood on the finger pricker or cap
- Always before someone else handles the finger pricker, for example, to assist you.

You need lint-free cloths in which you can wrap the whole finger pricker, water and 70 % isopropanol. Before you disinfect the finger pricker and the cap, you first have to clean them to remove blood and other dirt.

To avoid damaging the finger pricker and the cap:

- Do not dip the finger pricker and the cap in the fluids.
- Use only water for cleaning and only 70 % isopropanol for disinfecting.
- Remove the cap from the finger pricker (see 2).

**EN Instructions for use**

The Accu-Chek FastClix finger pricker is a device for simple, virtually pain-free collection of capillary blood from the fingertips. The finger pricker has 11 different penetration depths so that you can adjust the penetration depth to the texture of your skin. With the release button, you can prime the finger pricker and trigger a lancet in a single step. When a lancet is triggered, it penetrates the skin. You insert a lancet drum with 6 lancets into the finger pricker. You can obtain blood using a new, sterile lancet 6 times before the lancet drum needs changing.

These instructions for use feature the following 3 symbols:

- This symbol indicates a possible risk of injury or of damage to your health.
- This symbol draws attention to actions that could result in damage to the finger pricker.
- This symbol draws your attention to important information.

**The Accu-Chek FastClix finger pricker is intended for patient self-monitoring by an individual person only.** It must not be used to collect blood from more than one person as it does not incorporate any features to guard against cross-infection. There is a risk of infections being transmitted if the finger pricker is used by other people, even by family members, or if healthcare professionals use this finger pricker to obtain blood from different people. This finger pricker is therefore not suitable for professional use in healthcare facilities and institutions.

- Keep the finger pricker with all its components out of reach of children under 3 years. There is a risk of suffocation if small parts (e.g. caps or similar objects) are swallowed.

Only insert Accu-Chek FastClix lancet drums into the Accu-Chek FastClix finger pricker. Other lancets or lancet drums may impair the operation of the finger pricker or damage it permanently.

**Preparation**

- Wash your hands with warm water and soap before you obtain blood (see 1).
- Dry your hands thoroughly. This reduces contamination of the puncture site and stimulates blood flow.

**Inserting the lancet drum**

- You have to insert a lancet drum to be able to obtain blood with the finger pricker.
- Remove the cap from the finger pricker (see 2).

- Insert a new lancet drum – white end first – into the finger pricker (see 2).

The lancet drum is properly inserted when it is completely inside the finger pricker, and only the projecting edge protrudes from the finger pricker at the front (see 3).

You must not insert the lancet drum into the finger pricker and simultaneously press the release button or hold the finger pricker with the release button resting on a surface such as a table top. This could release a lancet and inadvertently cause injury.

- Place the cap back on again. The cut-out section at the edge of the cap must fit around the correspondingly shaped curvature with the penetration depth indicator (silver square) (see 4).

The cap is on correctly when it is pushed back as far as it will go, and you hear and feel it click into place.

The cap can only be correctly placed up to the stop position when the lancet drum is completely inside the finger pricker. Do not use any force to place the cap on as this might damage the cap and the finger pricker. Correct the position of the lancet drum instead.

Each lancet drum contains 6 lancets. The first lancet is now ready to be used. The window showing the number of available lancets shows 6; there are now 6 lancets available (see 5).

Do not remove the lancet drum before all 6 lancets have been used. The lancet drum cannot be re-inserted once it has been removed from the finger pricker.

**Setting the penetration depth**

You can set 11 penetration depths (0.5 to 5.5). The silver square (penetration depth indicator) on the finger pricker indicates the current penetration depth setting. The larger the number, the greater the penetration depth.

If you have no experience with this finger pricker, we recommend setting a low penetration depth, such as penetration depth 2.

- Rotate the cap until the desired penetration depth lines up with the indicator (see 6).

**Obtaining a blood drop**

In general, blood can be obtained from any finger. Certain fingers can be unsuitable if, for example, a skin or fingernail infection is present.

We recommend obtaining capillary blood from the sides of the fingertips as these areas are the least sensitive to pain.

Always use a new lancet when you obtain blood. In doing this, you reduce the risk of infection, and obtaining blood remains virtually pain free.

Only use the finger pricker when the cap is attached. Without the cap, the lancet penetrates too deeply and the puncture can become painful.

In individual cases, the lancet may protrude from the cap after obtaining blood. There is a risk of injury or infection if you or another person touches the protruding lancet. If the lancet protrudes from the cap, carefully set the finger pricker aside. Keep the finger pricker out of reach of other people and contact the Accu-Chek Customer Careline.

- Press the finger pricker firmly against the selected puncture site (see 7).
- Press the release button all the way down (see 8).

The lancet is released and it penetrates the skin. Massage the finger in the direction of the fingertip to encourage a blood drop to form.

The amount of blood that emerges depends on the penetration depth and the pressure used to hold the finger pricker against the skin.

If not enough blood emerges, apply more pressure on the finger pricker when you next obtain blood. If that is not sufficient, increase the penetration depth progressively in addition.

If too much blood emerges, decrease the penetration depth.

Test your blood glucose immediately after you obtained blood, according to the instructions for use for your blood glucose meter.

We recommend protecting the puncture site from contamination afterwards.

**Loading a new lancet**

If you have already used the finger pricker to obtain blood, you will need a new lancet the next time you obtain blood.

- Push the lever at the side of the finger pricker as far as it will go in direction A and then back in direction B (see 9).

The next lancet is now ready to be used.

The window next to the lever shows the new number of available lancets, which has decreased by one, for example, 5 instead of 6.

- Proceed as in the previous step each time you need to change the lancet.

Each time a new lancet is loaded, the number of available lancets shown in the window decreases by one, for example, from 4 to 3. When you have loaded the sixth and last lancet, the window shows 1.

**Replacing a used lancet drum**

When you have used the sixth and last lancet, you need to replace the used lancet drum with a new one.

- Remove the cap from the finger pricker (see 2).
- Remove the lancet drum from the finger pricker (see 10).

When a lancet drum has been used, a red stripe is visible on the white cylinder.

A used lancet drum has a built-in lock that stops you from using it again.

Used lancet drums can be disposed of in household waste if no other regulations apply in your country.

**Storing the finger pricker**

Do not store the finger pricker at extreme temperatures, for example, in a hot car. This can impair the operation of the finger pricker.

**Cleaning and disinfecting the finger pricker and cap**

To prevent infections from being transmitted, you must clean and disinfect the finger pricker and cap regularly:

- At least once a week
- When there is blood on the finger pricker or cap
- Always before someone else handles the finger pricker, for example, to assist you.

You need lint-free cloths in which you can wrap the whole finger pricker, water and 70 % isopropanol. Before you disinfect the finger pricker and the cap, you first have to clean them to remove blood and other dirt.

To avoid damaging the finger pricker and the cap:

- Do not dip the finger pricker and the cap in the fluids.
- Use only water for cleaning and only 70 % isopropanol for disinfecting.
- Remove the cap from the finger pricker (see 2).

**EN Instructions for use**

The Accu-Chek FastClix finger pricker is a device for simple, virtually pain-free collection of capillary blood from the fingertips. The finger pricker has 11 different penetration depths so that you can adjust the penetration depth to the texture of your skin. With the release button, you can prime the finger pricker and trigger a lancet in a single step. When a lancet is triggered, it penetrates the skin. You insert a lancet drum with 6 lancets into the finger pricker. You can obtain blood using a new, sterile lancet 6 times before the lancet drum needs changing.

These instructions for use feature the following 3 symbols:

- This symbol indicates a possible risk of injury or of damage to your health.
- This symbol draws attention to actions that could result in damage to the finger pricker.
- This symbol draws your attention to important information.

**The Accu-Chek FastClix finger pricker is intended for patient self-monitoring by an individual person only.** It must not be used to collect blood from more than one person as it does not incorporate any features to guard against cross-infection. There is a risk of infections being transmitted if the finger pricker is used by other people, even by family members, or if healthcare professionals use this finger pricker to obtain blood from different people. This finger pricker is therefore not suitable for professional use in healthcare facilities and institutions.

- Keep the finger pricker with all its components out of reach of children under 3 years. There is a risk of suffocation if small parts (e.g. caps or similar objects) are swallowed.

Only insert Accu-Chek FastClix lancet drums into the Accu-Chek FastClix finger pricker. Other lancets or lancet drums may impair the operation of the finger pricker or damage it permanently.

**Preparation**

- Wash your hands with warm water and soap before you obtain blood (see 1).
- Dry your hands thoroughly. This reduces contamination of the puncture site and stimulates blood flow.

**Inserting the lancet drum**

- You have to insert a lancet drum to be able to obtain blood with the finger pricker.
- Remove the cap from the finger pricker (see 2).

- Insert a new lancet drum – white end first – into the finger pricker (see 2).

The lancet drum is properly inserted when it is completely inside the finger pricker, and only the projecting edge protrudes from the finger pricker at the front (see 3).

You must not insert the lancet drum into the finger pricker and simultaneously press the release button or hold the finger pricker with the release button resting on a surface such as a table top. This could release a lancet and inadvertently cause injury.

- Place the cap back on again. The cut-out section at the edge of the cap must fit around the correspondingly shaped curvature with the penetration depth indicator (silver square) (see 4).

The cap is on correctly when it is pushed back as far as it will go, and you hear and feel it click into place.

The cap can only be correctly placed up to the stop position when the lancet drum is completely inside the finger pricker. Do not use any force to place the cap on as this might damage the cap and the finger pricker. Correct the position of the lancet drum instead.

Each lancet drum contains 6 lancets. The first lancet is now ready to be used. The window showing the number of available lancets shows 6; there are now 6 lancets available (see 5).

Do not remove the lancet drum before all 6 lancets have been used. The lancet drum cannot be re-inserted once it has been removed from the finger pricker.

**Setting the penetration depth**

You can set 11 penetration depths (0.5 to 5.5). The silver square (penetration depth indicator) on the finger pricker indicates the current penetration depth setting. The larger the number, the greater the penetration depth.

If you have no experience with this finger pricker, we recommend setting a low penetration depth, such as penetration depth 2.

- Rotate the cap until the desired penetration depth lines up with the indicator (see 6).

**Obtaining a blood drop**

In general, blood can be obtained from any finger. Certain fingers can be unsuitable if, for example, a skin or fingernail infection is present.

We recommend obtaining capillary blood from the sides of the fingertips as these areas are the least sensitive to pain.

Always use a new lancet when you obtain blood. In doing this, you reduce the risk of infection, and obtaining blood remains virtually pain free.

Only use the finger pricker when the cap is attached. Without the cap, the lancet penetrates too deeply and the puncture can become painful.

In individual cases, the lancet may protrude from the cap after obtaining blood. There is a risk of injury or infection if you or another person touches the protruding lancet. If the lancet protrudes from the cap, carefully set the finger pricker aside. Keep the finger pricker out of reach of other people and contact the Accu-Chek Customer Careline.

- Press the finger pricker firmly against the selected puncture site (see 7).
- Press the release button all the way down (see 8).

The lancet is released and it penetrates the skin. Massage the finger in the direction of the fingertip to encourage a blood drop to form.

The amount of blood that emerges depends on the penetration depth and the pressure used to hold the finger pricker against the skin.

If not enough blood emerges, apply more pressure on the finger pricker when you next obtain blood. If that is not sufficient, increase the penetration depth progressively in addition.

If too much blood emerges, decrease the penetration depth.

Test your blood glucose immediately after you obtained blood, according to the instructions for use for your blood glucose meter.

We recommend protecting the puncture site from contamination afterwards.

**Loading a new lancet**

If you have already used the finger pricker to obtain blood, you will need a new lancet the next time you obtain blood.

- Push the lever at the side of the finger pricker as far as it will go in direction A and then back in direction B (see 9).

The next lancet is now ready to be used.

The window next to the lever shows the new number of available lancets, which has decreased by one, for example, 5 instead of 6.

- Proceed as in the previous step each time you need to change the lancet.

Each time a new lancet is loaded, the number of available lancets shown in the window decreases by one, for example, from 4 to 3. When you have loaded the sixth and last lancet, the window shows 1.

**Replacing a used lancet drum**

When you have used the sixth and last lancet, you need to replace the used lancet drum with a new one.

- Remove the cap from the finger pricker (see 2).
- Remove the lancet drum from the finger pricker (see 10).

When a lancet drum has been used, a red stripe is visible on the white cylinder.

A used lancet drum has a built-in lock that stops you from using it again.











Used lancet drums can be disposed of in household waste if no other regulations apply in your country.

**Storing the finger pricker**

Do not store the



SV FI  
**ACCU-CHEK®**  
**FastClix**  
 Blodprovstagare  
 Lansettikynä

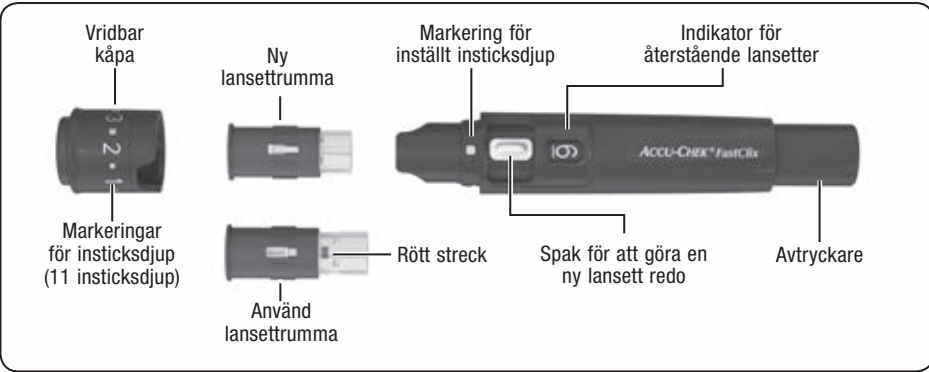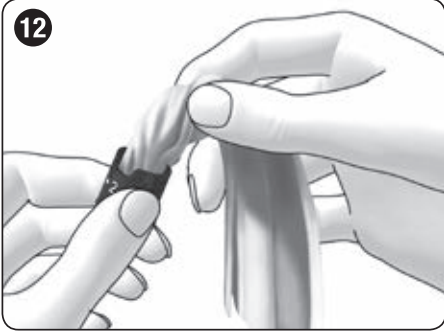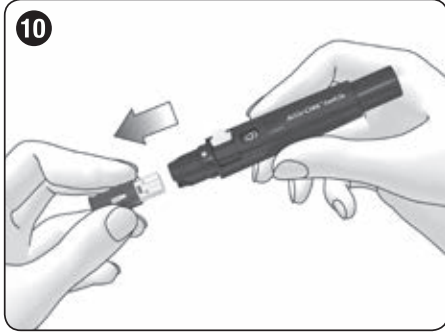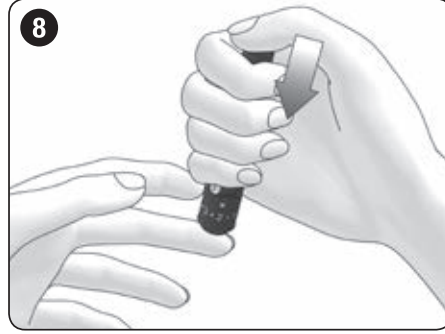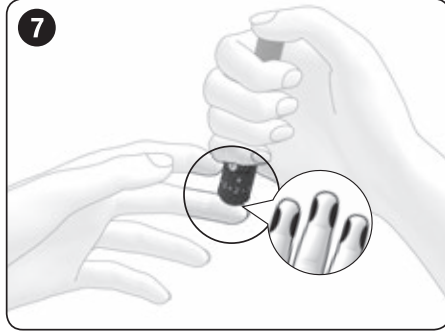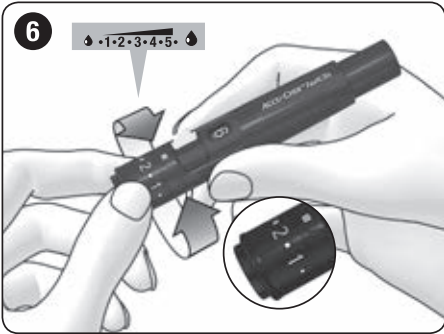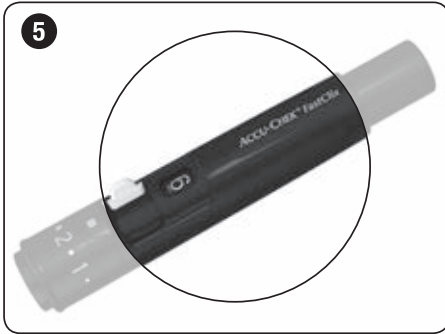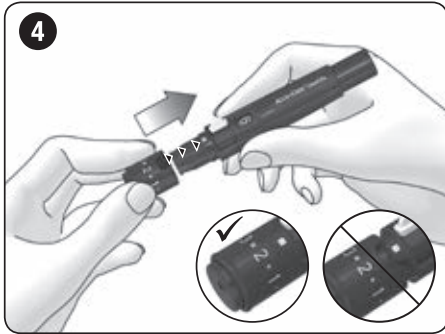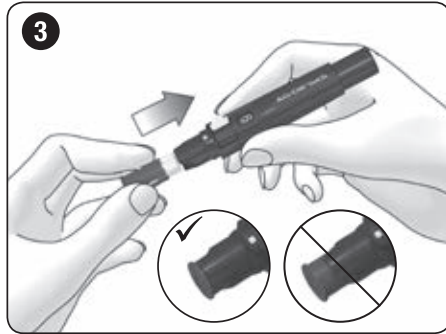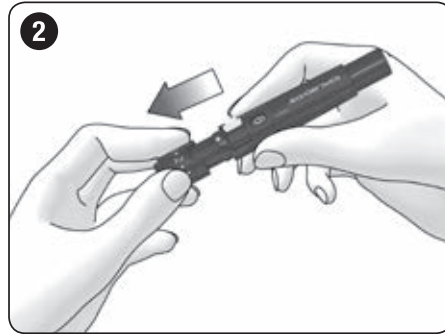

Lansetttrumman är rätt isatt när den är helt instucken i blodprovstagaren och det bara är den utstickande kanten som sticker ut fram till (se 1).  
 Du får inte samtidigt sätta in lansetttrumman i blodprovstagaren och trycka på avtryckaren eller stötdja blodprovstagaren med avtryckaren mot exempelvis en bordsskiva. Om du gör det kan en lansett matas ut och du kan sticka dig oavsiktligt.  
 Sätt på kåpan igen. Se till att inbuktningen i kåpans kant griper tag om utbuktningen med markeringen för insticksdjup (silverfärgad kvadrat) (se 2).  
 Kåpan är rätt påsatt när du har skjutit på den så långt det går och hört och känt att den har snäppt på plats.

Det går bara att sätta på kåpan utan problem om lansetttrumman är helt instucken i blodprovstagaren. Du får inte använda våld för att sätta på kåpan. Kåpan och blodprovstagaren kan skadas om du gör det. Rätta i stället till lansetttrummans låge.  
 Varje lansetttrumma innehåller 6 lansetter. Den första lansetten kan användas. Indikatorn för återstående lansetter visar 6, vilket betyder att du har 6 lansetter kvar (se 3).  
 Ta inte ut lansetttrumman förrän alla 6 lansetterna är använda. Det går inte att sätta tillbaka lansetttrumman när du väl har tagit ut den.

**Ställa in insticksdjupet**  
 Du kan ställa in 11 insticksdjup (0,5 till 5,5). Den silverfärgade kvadraten (markering för inställt insticksdjup) på blodprovstagaren anger det aktuella insticksdjupet. Ju högre siffra, desto större är insticksdjupet.  
 Om du inte har någon tidigare erfarenhet av den här blodprovstagaren rekommenderar vi att du ställer in ett lågt insticksdjup, till exempel 2.  
 Vid på kåpan tills önskat insticksdjup står mitt för markeringen (se 4).

**Ta blod**  
 I allmänhet kan du ta blod från vilket finger som helst. Vissa fingrar kan vara olämpliga, till exempel om du har en svår infektion i huden eller nageln. Vi rekommenderar att du tar kapillärblood från sidan av fingertopparna, eftersom de här ställena är minst känsliga för smärta.

Använd en ny lansett varje gång som du tar blod. Detta minskar inte bara infektionsrisken, utan ger även en så gott som smärtfri blodprovstagning.  
 Använd endast blodprovstagaren när kåpan är på. Utan kåpa tränger lansetten in för djupt och sticker en bordsskiva ont.  
 I såltsynia fall kan lansetten sticka ut genom kåpan efter blodprovstagning. Det finns risk för skada och infektion om du eller någon annan person rör den utstickande lansetten. Om en lansett sticker ut genom kåpan, ska du försiktigt lägga ifrån dig blodprovstagaren. Förvara blodprovstagaren oåtkomligt för andra personer och kontakta kundsupport.  
 Tryck blodprovstagaren hårt mot det valda stickstället (se 7).  
 Tryck in avtryckaren helt (se 8). Detta matar fram lansetten, så att den sticks in i huden.  
 Massera fingret i riktning mot fingertoppen för att på så sätt lättare få fram en bloddropp.  
 Mängden blod som kommer ut beror på vilket insticksdjup som ställs in och hur hårt du trycker blodprovstagaren mot huden.  
 Om du inte får fram tillräckligt med blod, ska du trycka blodprovstagaren hårdare mot huden vid nästa blodprovstagning. Om du fortfarande inte får fram tillräckligt med blod, ska du även öka insticksdjupet stegvis.  
 Om det kommer ut för mycket blod, ska du minska insticksdjupet.  
 När du har tagit blod med blodprovstagaren, behöver du en ny lansett för nästa blodprovstagning.  
 Skjut spaken på sidan av blodprovstagaren i riktning mot A så långt det går och därefter tillbaka i riktning mot B (se 9).  
 Nästa lansett kan nu användas.  
 Indikatorn bredvid spaken visar nu att det finns en lansett mindre kvar, till exempel 5 i stället för 6.  
 Följ anvisningarna i förra steget för att ladda lansetterna i tur och ordning.

Varje gång en ny lansett laddas minskar värdet i indikatorn för återstående lansetter med ett, till exempel från 4 till 3. När du har laddat den sjätte och sista lansetten, visar indikatorn 1.  
**Byta ut använd lansetttrumma**  
 När du har använt den sjätte och sista lansetten måste du byta ut den använda lansetttrumman mot en ny.  
 Ta bort kåpan från blodprovstagaren (se 2).  
 Dra ut lansetttrumman ur blodprovstagaren (se 10).  
 På en använd lansetttrumma syns ett rött streck på den vita cylindern.  
 Sätt in en ny lansetttrumma (se avsnittet *Sätta in lansetttrumman*).  
 En använd lansetttrumma har en inbyggd säkerhetsspår som gör att det inte går att sätta in den igen.  
 Du kan kassera använda lansetttrummor i hushållsavfallet om de lokala reglerna i ditt land inte föreskriver något annat.

**Förvara blodprovstagaren**  
 Förvara aldrig blodprovstagaren vid höga temperaturer, till exempel i en varm bil. Funktionen kan försämrans.  
**Rengöra och desinficera blodprovstagaren och kåpan**  
 För att förhindra infektionsspridning måste du rengöra och desinficera blodprovstagaren och kåpan regelbundet:  
 • minst 1 gång i veckan,  
 • om de har blivit nedsmutsade med blod,  
 • alltid innan någon annan person tar i blodprovstagaren för att till exempel hjälpa dig.  
 Du behöver luddfria trasor som du kan linda in blodprovstagaren fullständigt i, vatten och 70-procentig isopropanol.  
 Innan du desinficerar blodprovstagaren och kåpan måste du först rengöra dem för att ta bort blod och annan smuts.  
 För att inte skada blodprovstagaren och kåpan:  
 • Doppa aldrig ned blodprovstagaren eller kåpan i vätskorna.  
 • Använd endast vatten till rengöringen och 70-procentig isopropanol till desinficeringen.  
 • Ta bort kåpan från blodprovstagaren (se 2).  
 • Dra ut lansetttrumman ur blodprovstagaren (se 10).

**Mata fram en ny lansett**  
 När du har tagit blod med blodprovstagaren, behöver du en ny lansett för nästa blodprovstagning.  
 Skjut spaken på sidan av blodprovstagaren i riktning mot A så långt det går och därefter tillbaka i riktning mot B (se 9).  
 Nästa lansett kan nu användas.  
 Indikatorn bredvid spaken visar nu att det finns en lansett mindre kvar, till exempel 5 i stället för 6.  
 Följ anvisningarna i förra steget för att ladda lansetterna i tur och ordning.

**Rengöra**  
 Fukta en trasa lätt med vatten.  
 Torka av blodprovstagaren och kåpan noga, blodprovstagaren endast utvändigt (se 11) och kåpan både utvändigt och invändigt (se 12).  
 Torka blodprovstagaren och kåpan med en torr trasa.  
 Ta bort kåpan från blodprovstagaren (se 2).  
 Dra ut lansetttrumman ur blodprovstagaren (se 10).  
 På en använd lansetttrumma syns ett rött streck på den vita cylindern.  
 Sätt in en ny lansetttrumma (se avsnittet *Sätta in lansetttrumman*).  
 En använd lansetttrumma har en inbyggd säkerhetsspår som gör att det inte går att sätta in den igen.  
 Du kan kassera använda lansetttrummor i hushållsavfallet om de lokala reglerna i ditt land inte föreskriver något annat.

**Andra provtagningsställen**  
 Kapillärblood för blodsockertest måste inte tas från fingertoppen utan kan också tas från alternativa provtagningsställen (handflatan, underarmen, överarmen, vaden eller äret) – så kallad Alternative Site Testing (AST). För att du ska kunna ta kapillärblood från andra ställen på kroppen måste du byta ut den normala kåpan mot en specialkåpa, Accu-Chek FastClix AST-kåpan.  
**Beställningsinformation**  
 Kontakta kundsupport (se avsnittet *Kundsupport*) om du vill beställa en Accu-Chek FastClix-blodprovstagare, Accu-Chek FastClix-lansetttrummor eller en Accu-Chek FastClix AST-kåpa.  
**Senaste revidering**  
 2019-04  
**Kundsupport**  
 Sverige  
 Accu-Chek Kundsupport: 020-41 00 42  
 www.accu-chek.se  
 Finland  
 Kundtjänsttelefon: 0800 92066 (kostnadsfri)  
 www.accu-chek.fi

**Finland**  
 Kundtjänsttelefon: 0800 92066 (kostnadsfri)  
 www.accu-chek.fi

**Revidering**  
 2019-04

**Revidering**  
 2019-04

**Revidering**  
 2019-04

Observera, läs säkerhetsanvisningarna i bruksanvisningen som medföljer denna produkt.  
 Enbart för engångsbruk  
 Använd före  
 Steriliserat med bestrålning  
 Tillverkare  
 Artikelnummer  
 Batchkod  
 Globalt artikelnummer

Denna produkt uppfyller kraven i det europeiska direktivet 93/42/EEG om medicintekniska produkter.

Roche Diabetes Care GmbH  
 Sandhofer Strasse 116  
 68305 Mannheim, Germany  
 www.accu-chek.com

**Bruksanvisning**

Accu-Chek FastClix-blodprovstagaren är ett hjälpmedel som är enkelt att använda och som utvecklats för så gott som smärtfri provtagning av kapillärblood från fingertoppen. Blodprovstagaren har 11 olika insticksdjup så att du kan anpassa insticksdjupet till just din hud. Med avtryckaren kan du i ett enda steg ladda blodprovstagaren och mata fram en lansett. När lansetten matas fram sticks den in i huden. I blodprovstagaren sätter du in en lansetttrumma med 6 lansetter. Sedan kan du ta blod med en steril lansett 6 gånger, innan du behöver byta lansetttrumma.

I den här bruksanvisningen används följande 3 symboler:

- Den här symbolen indikerar att det finns risk för skada eller för påverkan på din egen hälsa.
- Den här symbolen indikerar hantering som kan leda till skador på blodprovstagaren.
- Den här symbolen indikerar viktig information.


**Accu-Chek FastClix-blodprovstagaren är endast avsedd för personligt bruk!**

Den får alltså endast användas för att ta blod från en och samma person.  
 Det finns risk för infektionsspridning om blodprovstagaren används av flera personer, även om de tillhör samma familj, eller om sjukvårdspersonal använder den här blodprovstagaren för att ta blodprov från mer än en person. Blodprovstagaren är därför inte avsedd för professionell användning inom offentliga sjukvårdsinrättningar.  
 Förvara blodprovstagaren inklusive alla ingående delar oåtkomligt för barn under 3 år. Det finns risk för kvävning om smådelar (till exempel kåpor eller liknande) svälles.

Sätt endast in Accu-Chek FastClix-lansetttrummor i Accu-Chek FastClix-blodprovstagaren. Andra lansetter eller lansetttrummor kan försämrans blodprovstagarens funktion eller förstöra blodprovstagaren.

**Förberedelser**  
 Tvätta händerna med varmt vatten och tvål innan du tar blod (se 1). Torka händerna noggrant. Detta garanterar att stickstället är rent, samtidigt som det stimulerar blodflödet.

**Sätta in lansetttrumman**  
 För att kunna använda blodprovstagaren för att ta blod måste du först sätta in en lansetttrumma.  
 Ta bort kåpan från blodprovstagaren (se 2).  
 Sätt in en ny lansetttrumma i blodprovstagaren – med den vita änden först (se 3).

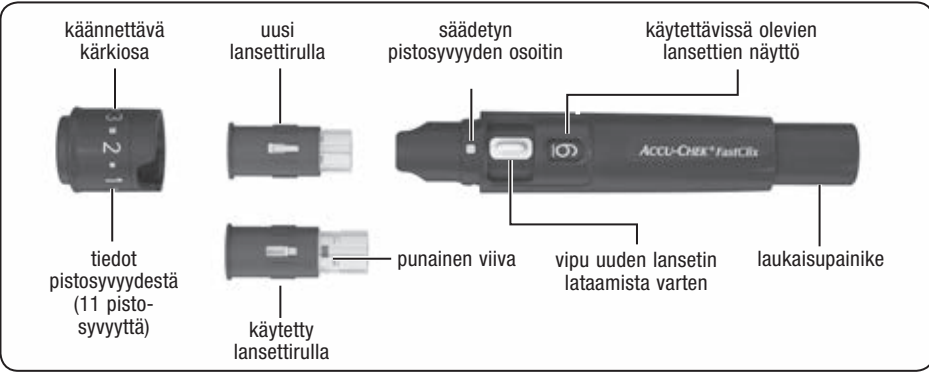

Aseta uusi lansettirulla – valkoinen osa edellä – lansettikynään (katso kuva 3).  
 Lansettirulla on asetettu oikein, kun se on kokonaan lansettikynän sisällä siten, että vain reuna-alue on näkyvässä (katso kuva 4).  
 Lansettirullaa ei saa asettaa lansettikynään samanaikaisesti, kun laukaisupainiketta painetaan, tai kun lansettikynän laukaisupainike on tuettuna esim. pöytää vasten. Muutoin lansetti voi mahdollisesti lauetta ja pistää tahattomasti.  
 Aseta karkiosa takaisin paikalleen. Karkiosan reunassa olevan loyven ja pistosvyyden osoittimen (hopeanvärinen neliö) vastaavan muotoisen kaaran on tällöin oltava kohdakkain (katso kuva 5).  
 Karkiosa on asetettu oikein, kun se on työnnetty vasteseen saakka ja loksahdan kuuluvasti ja tuntuavasti paikalleen.  
 Karkiosa työntyy helposti vasteseen asti vain, jos lansettirulla on täysin lansettikynän sisällä. Älä käytä voimaa asettaessasi karkiosaa paikalleen. Muutoin karkiosa ja lansettikynä saattavat vahingoittua. Korjaa siten tarvittaessa vain lansettirullan asentoa.  
 Jokainen lansettirulla sisältää 6 lansettia. Ensimmäinen lansetti on nyt käyttövalmis. Käytettävissä olevien lansettien määrän ilmoittavassa näytössä näkyy luku 6, mikä merkitsee, että käytettävissä on nyt 6 lansettia (katso kuva 6).  
 Älä poista lansettirullaa ennen kuin kaikki 6 lansettia on käytetty. Lansettirullaa ei voi asettaa takaisin paikalleen, jos se on otettu kerran irti.  
**Pistosvyyden säätäminen**  
 Valittavana on 11 pistosvyyttä (0,5–5,5). Lansettikynässä oleva hopeanvärinen neliö (säädetyt pistosvyyden osoitin) osoittaa säädetyt pistosvyyden. Mitä suurempi tämä luku on, sitä suurempi on myös pistosvyyttä.  
 Jos et ole käyttänyt tätä lansettikynää aikaisemmin, suosittelemme, että aloitat pienellä pistosvyydellä, esim. 2:lla.  
 Kierrä karkiosaa, kunnes haluamasi pistosvyyden osoittimen kohdalla (katso kuva 6).  
**Verinäytteen ottaminen**  
 Verinäyte voidaan ottaa periaatteessa mistä sormesta tahansa. Sormi, jonka ihossa tai

kynessä on infektiota, ei sovellu verinäytteen ottoon. Kapillaariverinäyte on suositeltavaa ottaa sormenpään sivulta, sillä näissä pistoskohdissa kiputuntemus on vähäisintä.  
 Aina kun otat verinäytteen, käytä uutta lansettia. Tämä sekä vähentää infektiovaaraa että auttaa myös ottamaan verinäytteen lähes kivuttomasti.  
 Käytä lansettikynää vain karkiosan ollessa paikallaan. Ilman karkiosaa lansetti työntyy liian syväälle ja pisto saattaa aiheuttaa kipua.  
 Yksittäispainikkeista lansetti saattaa työntyä ulos karkiosasta verinäytteen oton jälkeen. Jos kosketat tai joku toinen henkilö koskettaa ulos työntynyttä lansettia, on olemassa loukkaantumisaara ja infektioiden leviämisen riski. Jos lansetti on työntynyt ulos karkiosasta, aseta lansettikynä varovasti sivuun. Pidä lansettikynä poissa muiden henkilöiden ulottuvilta ja ota yhteyttä asiakaspalveluun.  
 Paina lansettikynä napakasti valitsemaasi pistoskohtaa vasten (katso kuva 7).  
 Paina laukaisupainike pohjaan saakka (katso kuva 8).  
 Tämä laukaisee lansetin, joka työntyy ihoon.  
 Hiero sormea sormenpäätä kohti edistääksesi veripisaran muodostumista.  
 Muodostuvan veripisaran suuruus riippuu pistosvyydestä ja siitä, kuinka voimakkaasti painat lansettikynää ihoa vasten.  
 Jos verta ei muodostu riittävästi, paina lansettikynää napakkammin ihoa vasten seuraavan verinäytteen oton yhteydessä. Jos verta ei edelleenkään tule riittävästi, suurena myös pistosvyyttä asteittain.  
 Jos verta tulee liikaa, pienennä pistosvyyttä.  
 Mittaa verensokeri heti verinäytteen oton jälkeen verensokerimittarisi käyttöohjeen mukaisesti.  
 Epäpuhtauksien välttämiseksi suosittelemme pistoskohdan suojaamista verinäytteen oton jälkeen.  
**Uuden lansetin lataaminen**  
 Jos olet ottanut verinäytteen lansettikynän avulla, tarvitset seuraavaa verinäytteenottoa varten uuden lansetin.  
 Liikuta lansettikynän sivussa oleva vipu vasteseen asti suuntaan A ja sitten suuntaan B (katso kuva 9).  
 Seuraava lansetti on nyt käyttövalmis.

Vivun vieressä oleva näyttö osoittaa käytettävissä olevien lansettien määrän pienentyneen yhdellä, esim. 5 aikaisemmin 6:n sijaan.  
 Toimi edellä kuvatulla tavalla joka kerta, kun vaihdat lansetin.  
 Jokaisen lansetin vaihdon jälkeen näyttö osoittaa käytettävissä olevien lansettien määrän pienentyneen yhdellä, esim. 5 aikaisemmin 4:n sijaan. Kun olet ladannut kuudennen ja siten viimeisen lansetin käyttöön, näytöllä on luku 1.  
**Käytetyn lansettirullan vaihtaminen**  
 Kun olet käyttänyt kuudennen ja siten viimeisen lansettin, käytetyn lansettirullan tilalle on vaihdettava uusi lansettirulla.  
 Vedä lansettikynän karkiosa irti (katso kuva 2).  
 Poista lansettirulla lansettikynästä (katso kuva 10).  
 Käytetyn lansettirullan valkoisessa syinterissä näkyy punainen viiva.  
 Aseta uusi lansettirulla paikalleen (katso kohta *Lansettirullan asettaminen*).  
 Lansettirullassa on suojausmekanismi, joka estää käytetyn lansettirullan asettamisen uudelleen paikalleen.  
 Käytetyt lansettirullat voidaan hävittää talousjätteen mukana, mikäli maassasi voimassa olevat määräykset eivät toisin määrää.  
**Lansettikynän säilytys**  
 Älä säilytä lansettikynää korkeissa lämpötiloissa, kuten esim. kuumassa autossa. Se voi aiheuttaa toimintahäiriöitä.  
**Lansettikynän ja karkiosan puhdistus ja desinfiointi**  
 Infektioiden leviämisen välttämiseksi lansettikynä ja karkiosa on puhdistettava ja desinfiotava säännöllisesti:  
 • vähintään kerran viikossa  
 • jos se on tahrilintunut veren  
 • aina ennen kuin toinen henkilö koskettaa lansettikynää esim. auttaessaan sinua.  
 Tätä varten tarvitset nukkaamattoman liinan, johon lansettikynä voidaan kokonaan kääriä, vettä ja 70-prosenttista isopropanolia.  
 Veren ja muiden tahrojen poistamiseksi desinfiointi.

Lansettikynän ja karkiosan vahingoittumisen välttämiseksi:  
 • Älä upota lansettikynää ja karkiosaa nesteeseen.  
 • Käytä puhdistuksessa vain vettä ja desinfiointissa vain 70-prosenttista isopropanolia.  
 Vedä lansettikynän karkiosa irti (katso kuva 2).  
 Poista lansettirulla lansettikynästä (katso kuva 10).  
**Puhdistus**  
 Kostuta liina kevyesti vedellä.  
 Pyyhi lansettikynä ja karkiosa perusteellisesti: lansettikynä vain ulkopuolelta (katso kuva 11) ja karkiosa ulko- ja sisäpuolelta (katso kuva 12).  
 Kuivaa lansettikynä ja karkiosa kuivalla liinalla.  
**Desinfiointi**  
 Kostuta liina huolellisesti 70-prosenttisellä isopropanolilla.  
 Pyyhi lansettikynää ja karkiosaa kumpaakin 2 minuutin ajan perusteellisesti: lansettikynä vain ulkopuolelta ja karkiosa ulko- ja sisäpuolelta.  
 Aseta karkiosa takaisin paikalleen.  
 Kääri lansettikynä kokonaan liinan sisään.  
 Anna lansettikynän olla liinassa kääritynä 8 minuutin ajan.  
 Ota lansettikynä pois kääreestä ja anna sen kuivua ilmassa paikassa.

**Vaihtoehtoiset näytteenottopaikat verinäytteen ottoon**  
 Verensokerin mittausta varten kapillaariverinäytteitä voidaan ottaa sormenpään lisäksi myös muista kehonosista (kämmenestä, kynnärvarresta, olkavarresta, pohkeesta tai reidestä). Näistä paikoista otettuihin verinäytteisiin perustavasta mittauksesta käytetään nimitystä vaihtoehtoisia näytteenottopaikoista mittaaminen (englanniksi: Alternative Site Testing, lyhennettynä: AST). Jotta verinäyte voidaan ottaa muista kehonosista, tavallisen karkiosan tilalle on vaihdettava erityinen karkiosa, Accu-Chek FastClix AST-karkiosa.

**Tilustiedot**  
 Jos haluat tilata Accu-Chek FastClix -lansettikynän, Accu-Chek FastClix -lansettirullaa tai Accu-Chek FastClix AST-karkiosan, ota yhteyttä asiakaspalveluun (katso kohta *Asiakaspalvelu*).

**Viimeisin päivitys**  
 2019-04  
**Asiakaspalvelu**  
 Suomi  
 Asiakaspalvelupuhelin: 0800 92066 (maksuton)  
 www.accu-chek.fi

Vaara, tutustukaa tuotteen käyttöohjeeseen oleivin turvallisuuteen liittyviin huomautuksiin.  
 Käytä vain kerran  
 Käytettävä viimeistään  
 Steriloitu säteilyttämällä  
 Valmistaja  
 Tuotenumero  
 Eräkoodi  
 Maailmanlaajuinen kauppatavaranimero  
 Tämä tuote täyttää lääkinnällisiä laitteita koskevan Euroopan unionin direktiivin 93/42/EY vaatimukset.

Roche Diabetes Care GmbH  
 Sandhofer Strasse 116  
 68305 Mannheim, Germany  
 www.accu-chek.com